SpaceStation / SpaceCover / SpaceCom

Service Manual



Revision 2.0 English



This Service Manual is valid for:	Designation Part No.	
	SpaceStation without SpaceCom)
	SpaceStation with SpaceCom 0871 3142	<u>)</u>
	SpaceCover standard	7
	SpaceCover comfort	5
This Service Manual is available under	Designation Part No.	
the following part number:	Service Manual SpaceStation, English 8713 9220)
Languages of this Manual	The Service Manual for this unit can be supplied in the following languages:	
	Designation Part No.	
	German)
	English (US)	J
	French)

Table of Contents

Important Preliminary Remarks	Service Work	Page	0 - 5
	Technical Safety Checks	Page	0 - 5
	Current Versions	Page	0 - 5
	Revision Service	Page	0 - 5
	Responsibility of the Manufacturer	Page	0 - 6
	Quality Management	Page	0 - 6
	Checks and Repair	Page	0 - 6
	Notes on ESD	Page	0 - 6
	Spare Parts and Test Equipment	Page	0 - 7
	Setting Off	Page	0 - 7
	List of Abbreviations	Page	0 - 9
Contact Persons	Technical Training	Page	0 - 11
	Entry for Technical Training	Page	0 - 11
	Ordering of Spare Parts and Test Equipment	Page	0 - 11
	Service Hotline	Page	0 - 11
	Return of Spare Parts and Test Equipment	Page	0 - 11
	Safety Officer	9 -	
	(§ 30 MPG)	Page	0 - 11
	Translation	Page	0 - 11
System Overview	Description	Page	1 - 1
	System Overview	Page	1 – 1
	Physical construction	Page	1 - 2
	Function	Page	1 - 6
	Unit Software	Page	1 - 12
	Service Program	Page	1 - 14
	SpaceOnline	Page	1 - 17
	(only required with SpaceStation with SpaceCom)	_	1 - 17
	BCCshow	Page	1 - 17
	Technical Data	Page	1 - 13
	Options	_	
	Accessories	Page Page	1 - 21 1 - 21
Unit Diagnasis / Calibration	General	Dogo	2 1
Unit Diagnosis / Calibration	Alarms and Error Codes	Page	2 - 1 2 - 3
		Page	2 - 3 2 - 4
	The Most Important Error Modes Device Check	Page	
	Device Check SpaceCom	Page	2 - 4
	Trouble Shooting	Page Page	2 - 5 2 - 6
	Housic Shooting	rage	2 - 0
Disassembly / Assembly SpaceStation	General on the SpaceStation without SpaceCom	Page	3 - 1
	Tube Guide	Page	3 - 3
	Pole Clamp Guide	Page	3 - 4
	Housing Back Panel	Page	3 - 5
	Device Bracket	Page	3 - 6
	Power Supply	Page	3 - 7
	Release Button	Page	3 - 11
	Interface Board / Module Lock	Page	3 - 12
	Housing	Page	3 - 20
	Assembly / Installation	Page	3 - 21
		-	

Table of Contents

	Checks after Repair	Page	3 - 24
Disassembly / Assembly SpaceCom	General on the SpaceStation with SpaceCom	Page	4 - 1
, , , ,	Battery Module	Page	4 - 3
	W-LAN Module	Page	4 - 4
	Tube Guide	Page	4 - 5
	Pole Clamp Guide	Page	4 - 6
	Housing Back Panel	Page	4 - 7
	Device Bracket	Page	4 - 10
	SPCO PCB	Page	4 - 11
	Further Disassembly	Page	4 - 12
	Assembly / Installation	Page	4 - 13
	Checks after Repair	Page	4 - 14
Disassembly / Assembly SpaceCover	General	Page	5 - 1
,, , , ,	Battery Compartment Cover	Page	5 - 3
	Battery Module	Page	5 - 4
	(Only SpaceCover comfort)	Page	5 - 4
	Handle	Page	5 - 5
	Housing Upper Part	Page	5 - 6
	Housing Back Panel	Page	5 - 8
	Loudspeaker	Page	5 - 9
	(Only SpaceCover comfort)	Page	5 - 9
	Processor PCB	Page	5 - 10
	(Only SpaceCover comfort)	Page	5 - 10
	Assembly / Installation	Page	5 - 11
	Checks after Repair	Page	5 - 12
Servicing the Unit	Cleaning	Page	6 - 1
Servicing the Sint	Servicing the Battery	Page	6 - 1
	Servicing the battery	ruge	0 1
Technical Safety Check (TSC)		Page	7 - 1
Procedural Instructions on the TSC	Visual Inspection	Page	8 - 1
	Electrical Safety		
	according to IEC/EN 60601-1		
	or VDE 0750 and VDE 0751	Page	8 - 1
	Functional Inspection	Page	8 - 4
Test Equipment and Special Tools	Test Equipment	Page	9 - 1
	Special Tools	Page	9 - 1
Spare Parts List		Page	10 - 1
Revision Documentation	Description of Version	Page	11 - 1
	Version List of the Individual Pages	Page	11 - 1
Index		Page	12 - 1

Service Work

The present manual is for your information only. The possession of this manual does not authorize the performance of service work. Service tasks may only be executed by persons, who

- have received appropriate training on the system from B Braun
- are included in the revision service
- possess the necessary test equipment and mechanical aids,
 and
- fulfill the personal requirements (training and knowledge).

The user is obliged to perform or to have performed the Technical Safety Checks on those medial products for which these checks have been prescribed by the manufacturer and to carry them out according to the indications of the manufacturer as well as the generally approved technical standards while adhering to the periods stated (§ 6 MP BetreibV).

B. Braun also recommends training on the Technical Safety Checks, or to perform at least the steps indicated in the current version of the manual, as:

- the TSC requires that the instructions in the manuals are observed
- the manuals are a reference for measurements
- depending on the unit type, the Service Program must be called which may lead to a dangerous unit condition in case of inappropriate operation. Furthermore, a special service connector may be necessary.

This manual version corresponds to the state when the manual was written. B Braun reserves the right to make technical modifications. The state of the revision is indicated by the index number in the footer of every page.

The possession of this manual does not automatically mean inclusion in the revision service. You will be included in the revision service after:

- technical training by B. Braun Melsungen or
- a written order placed with the sales department of B. Braun (fee required).

Technical Safety Checks

Current Versions

Revision Service

0

Important Preliminary Remarks

Responsibility of the Manufacturer

The manufacturer, person who assembles, installs or imports the device can only be held responsible for safety, reliability and performance if

- mounting, enhancements, new settings, changes or repairs are carried out by duly authorized persons,
- the electrical installation in the corresponding room meets the requirements of the VDE 0107, VDE 0100 part 710 or IEC 60364-7-710 and the national standards,
- the device is used in accordance with the instructions for use and the Service Manual,
- the Technical Safety Checks are performed at regular intervals,
- a current manual which corresponds to the revision state is used when carrying out maintenance, repair and service,
- the service technician takes part in the revision service,
- the technician has participated in a technical training course for the specific B. Braun unit.

B. Braun is certified in accordance with DIN EN ISO 9001 and ISO 13485. This certification also includes maintenance and service.

The unit has the CE label. The CE label confirms that the device corresponds to the "Directive of the Council for Medical Products 93/42/EC" of June 14, 1993.

Training may only be performed by B. Braun. The possession of the manual does not authorize the performance of repairs. The instructions on electrostatic sensitive components (ESD standards) must be observed.


After repair a device check or diagnosis is to be carried out.

Semiconductors can be destroyed by electrostatic discharge. Especially MOS components can be damaged by interference from electrostatic fields, even without discharge via contact. This type of damage is not immediately recognizable. Unit malfunctions can even occur after a longer period of operation.

Quality Management

Checks and Repair

Notes on ESD

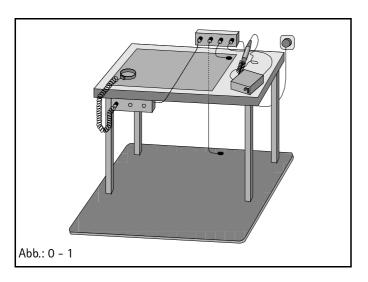

Spare Parts and Test Equipment

Setting Off

Each workstation must be equipped according to the recommendations with the necessary static protective measures, if ESD components or boards are handled.

Each workstation must be equipped with a conductive table surface. The conductive surface, the soldering iron or the soldering stations must be grounded via protective resistors.

Chairs must be of antistatic design. The floor or floor mats should be of electrically conductive material.

Personnel must wear conductive wristbands which are connected to a central ground potential via protective resistors, e.g. the ground contact of a wall outlet. Furthermore it is recommended that personnel wear cotton clothing and electrically conductive shoes to prevent electrostatic charge.

Only use original spare parts from the manufacturer. Do not tamper with assembly groups which can only be exchanged completely. The spare parts required are listed in Section 9.

Service personnel are responsible for the calibration of their test equipment. Original test equipment can be calibrated at the works of B. Braun. Further information is available upon request.

Additional notes and warnings are set off as follows:

Note

Is used for additional or special notes concerning information and working steps.

CAUTION

Is used for working steps which may result in damage to the unit, system or to a connected device.

WARNING

IS USED FOR WORKING STEPS WHICH MAY RESULT IN PERSONAL INJURY.

References to chapters are shown as follows

(see "Setting Off" → pg. 0 - 8)

References to figures and tables are shown as follows

Fig.: 2 - 3 or Table 2 - 1

O Important Preliminary Remarks

References to item numbers in figures are shown as follows (Fig.: 1 - 1 / Item 1)

In this case "Fig.: 1 - 1" is the figure number and "Item 1" the item number within the figure.

When the Service Manual is stored as pdf-file, these references are displayed green. Click with the mouse button on a reference to jump to the corresponding source.

Menu commands are described as:

Menu *File*.

List of Abbreviations

Abbreviations which are not generally known, but are used in this manual, are listed below.

CAN Controller Area Network CE Communauté Européenne

CS Calibration Step

DIN Deutsche Industrie Norm

(German Industrial Norm)

ΕN European Norm

ESD Electrostatic Discharge FuP **Function Microprocessor** IEC

International Electrotechnical

Commission

International Organization for IS0

Standardization,

ISP Infusomat® Space

ISPS Infusomat® Space (Silicon) **ISPP** Infusomat® Space, (PVC) KuP Monitoring Microprocessor LCD Liquid Crystal Display

MOS Short name of the following

company:

MOS Technology, Inc.

(Commodore Semiconductor

Group)

PCA Patient Controlled Analgesia

PSP Perfusor® Space SP Space (System) SPC SpaceCover

SPCC SpaceCover comfort **SPCS** SpaceCover standard

SPCO SpaceCom SPCT SpaceControl SPS SpaceStation TS Trouble Shooting UTS **Unit Test Step**

TSC **Technical Safety Check**

0

Important Preliminary Remarks

TEMP Temperature

VDE Verband der Elektrotechnik,

Elektronik und

Informationstechnik e.V. (German Association of

engineers)

0 - 10 SpaceStation, 2.0 gb

Technical Training Via local representative.

Entry for Technical Training Application for a technical training course must be made via the

responsible representative.

Ordering of Spare Parts and Test Equipment Please contact your local B. Braun subsidary.

International Technicians (Intercompany)

Nadja Machal

Fax: +49 5661 / 75 -47 89 e-mail: nadja.machal@bbraun.com

Service Hotline Karl Tippel, Tanja Kördel

Phone: +49 5661 / 71 - 35 25 Fax: +49 5661 / 71 - 35 26 e-mail: karl.tippel@bbraun.com e-mail: tanja.koerdel@bbraun.com

Return of Spare Parts and Test EquipmentB. Braun Melsungen AG

Schwarzenberger Weg 73-79 Wareneingang Werk C 34 212 Melsungen

Germany

Safety Officer Dr. Ludwig Schütz

(§ 30 MPG) e-mail: ludwig.schuetz@bbraun.com

Translation PAS GmbH, Brückner GmbH, Germany

O Contact Persons

For your information:	


0 - 12 SpaceStation, 2.0 gb

Description

The modular Space system is destined for treatment of an individual patient. It is designed for application on intensive care units and in the surgery. The Space system is operated in hospitals mainly by doctors and medically trained personnel.

The SpaceStation forms together with the SpaceCover an organization system. The Space system infusion and infusion syringe pumps are attached and connected to the SpaceStation.

System Overview

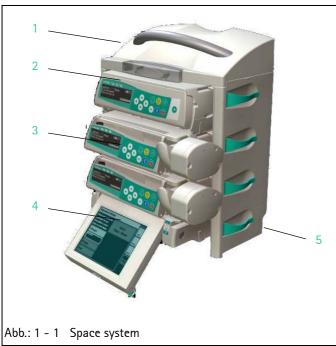

Legende zu Abb. 1 - 1: ItemDesignation

- 1 SpaceCover
- 2 Infusion pump Infusomat® Space
- 3 Infusion syringe pump Perfusor® Space
- 4 SpaceControl
- 5 SpaceStation

The Space system is a modular design of modern infusion technology for stationary, mobile or private use. The key modules and their connection to the peripheral devices are shown in Abb.: 1 -

All the pump types, Perfusor® Space, Infusomat® Space and Infusomat® Space P, as well as the other devices of the system are of modular design. Up to three pumps can be connected together using L rails on the bottom of the unit and grooves on the top. They can then be fastened to a drip stand or appropriate rail using the pole clamp.


The SpaceControl module can be used to extend operation. One single pump can be inserted onto this module. The pump is then connected via connectors to the module.

The SpaceStation module allows the set-up of a complete pump system with up to 24 pumps. Up to four pumps can be installed in every SpaceStation. The pumps are supplied with power via the integrated power supply and the built-in connectors. The pumps are connected to the optional SpaceCom via these connectors. SpaceControl can also be integrated into the system.

Up to six SpaceStations can be set-up as a column with a total of 24 pumps. SpaceStation placed next to each other can be connected via special connecting cables, if the maximum number of 24 pumps in maximum three columns is not exceeded.

SpaceCover Standard or SpaceCover Comfort forms the top of each column. Alarms are signalled by a row of LEDs and a loud-speaker in the SpaceCover Comfort.

Physical construction

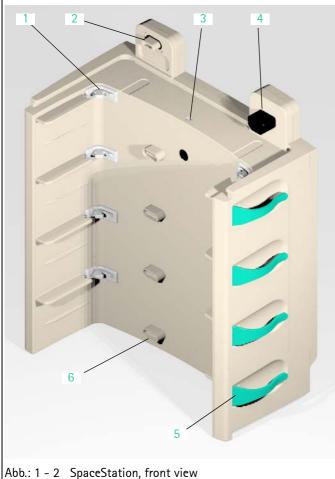

Legende zu Abb. 1 - 2:

Item Designation

- Pump lock 1
- Connector "F5" 2
- Module lock 3
- 4 Connector "F1B" (mains output)
- 5 Tube guide
- Connector "F2D"

SpaceStation without SpaceCom

The SpaceStation consists of the housing and the housing back panel.

The power supply with power connector and power socket for another SpaceStation is installed on the right side in the housing.

The pumps in the SpaceStation are locked by 8 locking claws which are built in the housing as subsystem "module locking". This subsystem also contains the locking for a SpaceCover or another SpaceStation.

The pump is released with a release button on the left side of the housing.

The pumps are connected to the system via the connectors "F2A" to "F2D" (from top to bottom) which are integrated in the interface board, while the SpaceStation is connected to other modules via the connectors "F3", "F4" and "F5". The interface board with connectors is installed on the left side in the housing.

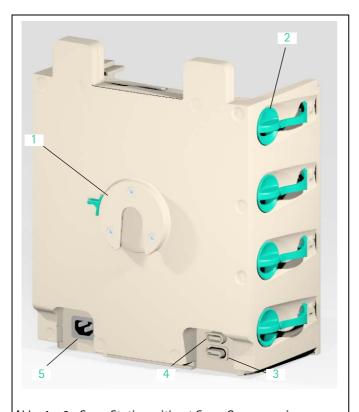


Abb.: 1 - 3 SpaceStation without SpaceCom, rear view

Legende zu Abb. 1 - 3:

ItemDesignation

- 1 Pole clamp guide
- 2 Release button
- 3 Connector "F3"
- 4 Connector "F4"
- 5 Connector "F1A" (power input)

Four tube guides are clipped in on the left and right housing side.

A pole clamp guide for attachment and locking of the pole clamp is located at the housing back panel.



Abb.: 1 - 4 SpaceStation with SpaceCom, rear view

Legende zu Abb. 1 - 4:

ItemDesignation


- 1 Fan grid
- 2 Battery compartment with battery compartment cover
- 3 Connector cover
- 4 W-LAN cover for W-LAN- module

SpaceStation with SpaceCom

The physical construction of the SpaceStation front side with SpaceCom corresponds exactly to the physical construction of the SpaceStation without SpaceCom. The only visible difference is to be found on the right side and the rear side.

Interfaces and a status- and alarm LED are located on the right-hand side behind the connector cover and the W-LAN cover. A battery module can be inserted in the battery compartment on the rear side. The opening is covered by the battery compartment cover.

A fan grid with fan is fitted on the left side of the battery compartment.

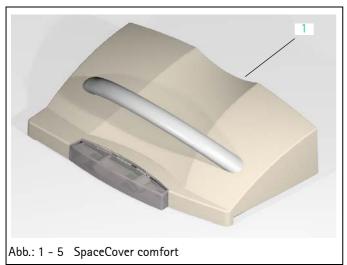

Legende zu Abb. 1 - 5: ItemDesignation

1 SpaceCover comfort

SpaceCover standard

The SpaceCover standard housing consists mainly of the bottom part and the upper part. A SpaceStation can be transported with the carrying handle.

SpaceCover comfort

The physical construction of the SpaceCover comfort corresponds to that of the SpaceCover standard. In addition to the SpaceCover standard the following subsystems are installed in the SpaceCover comfort:

- The battery module (as option)
 It is inserted from below in the battery compartment of the housing bottom part. The opening is covered by the battery compartment cover.
- Status and alarm display line installed at the front of the housing upper part.
- Operating and status display field at the left side of the housing upper part.

Function


SpaceStation without SpaceCom


The SpaceStation supplies the inserted pumps as well as a Space-Control connected to the pumps with the required voltage. The voltage is generated by the integrated power supply, which is connected to the supply mains via the connector "F1A" and a mains cable or an upstream SpaceStation. The mains voltage is transmitted via the internal cabling to connector "F1B" for connection of a further SpaceStation.

The pumps are connected with connectors "F2A" to "F2D" to the SpaceStation. These connectors provide the voltage supply, distribute the addresses in the Space system via a serial interface, transfer data via a bus system (CAN bus) and transmit a staff call, which may be pending.

Another SpaceStation inserted on top or a SpaceCover is integrated in the Space system via connector "F5".

Connector "F4" is used for connecting another SpaceStation or further SpaceStation columns. An upstream SpaceStation or an upstream SpaceStation column as well as accessories that may be connected, such as a PCA button or a service connector, are connected to connector "F3".

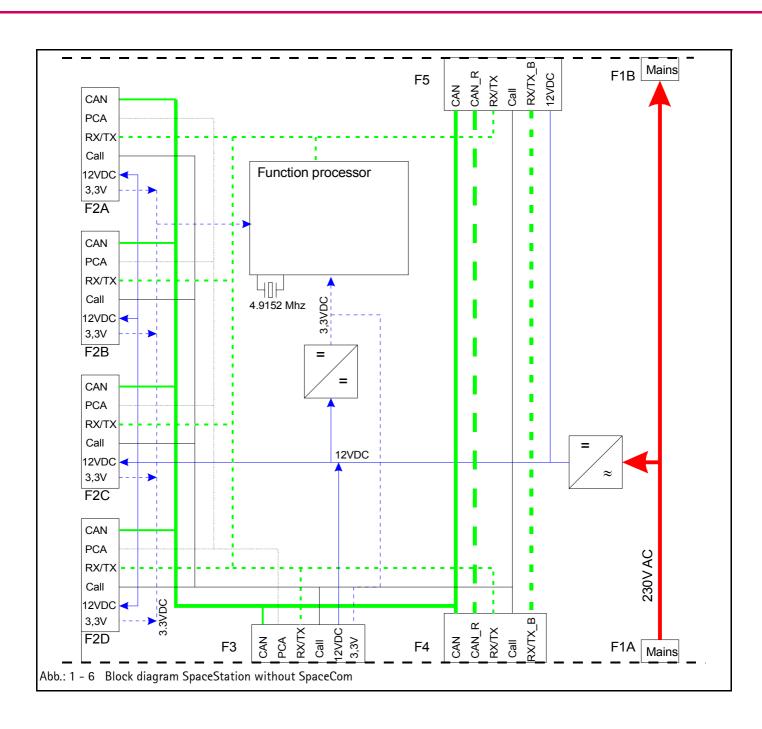

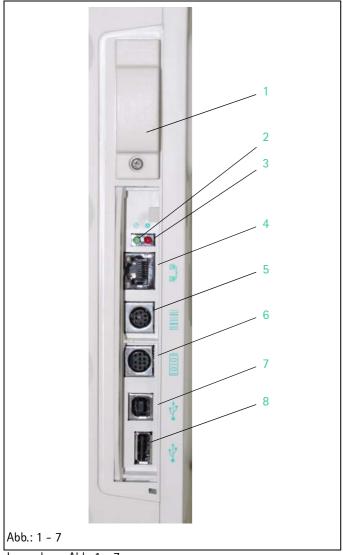

Legende zu Abb. 1 - 7:


ItemDesignation

- 1 W-LAN slot
- 2 Status indicator (green)
- 3 Alarm indicator (red)
- 4 Ethernet (RJ 45) connection
- 5 PS/2 connection for barcode reader
- 6 RS232 connection
- 7 USB connection (slave)
- 8 USB connection (master)

SpaceStation with SpaceCom

The SpaceCom provides further interfaces and connections, e.g. for a barcode scanner. These interfaces are depicted in Abb.: 1-7.

The battery module (option) serves to maintain the functions of the SpaceStation in case of a voltage failure. An independent circuit in the battery module monitors the battery cells and controls their charge condition. The SpaceStation must be covered with a SpaceCover comfort with inserted battery module to guarantee function of the battery module in the SpaceStation with SpaceCom.

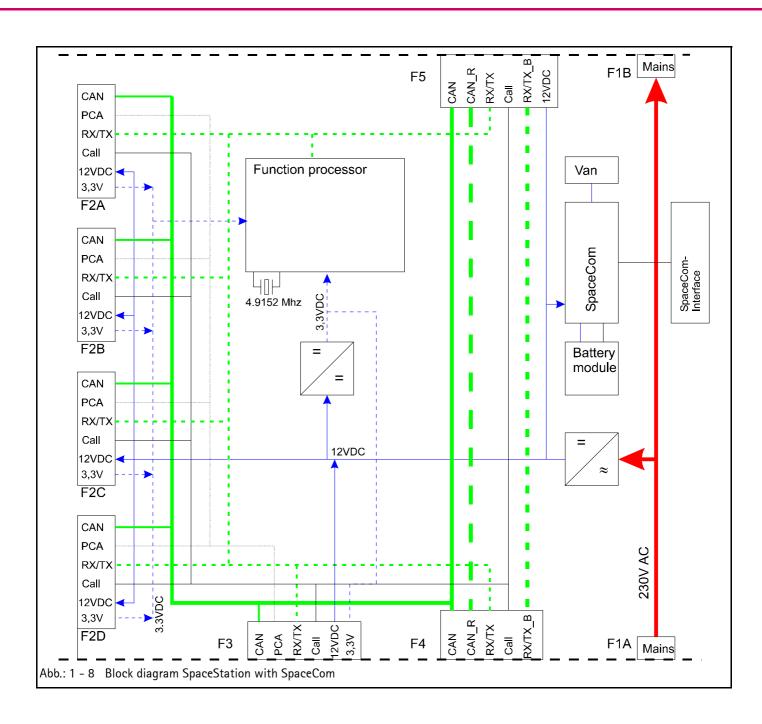

SpaceCover

The SpaceCover is connected to a SpaceStation with connector "D1"

With the SpaceCover standard this connector is used to connect the CAN bus and a serial interface.


In the SpaceCover comfort this connector provides the voltage supply, transmits a unique address in the Space system via a serial interface, transfers data via a bus system (CAN bus) and transmits a staff call, which may be pending.

The internal electronics controls the operator and status display field, the loudspeaker as well as the status and alarm display line. The status and alarm display line displays the collected messages of the pumps in a column. The brightness is controlled dependent on the environment.


The voltage supplied is converted to the internal voltages required through a voltage transforming and monitoring circuit on the processor PCB.

An independent circuit in the battery module monitors the battery cells and controls their charge condition.

1 - 10 SpaceStation, 1.0 gb

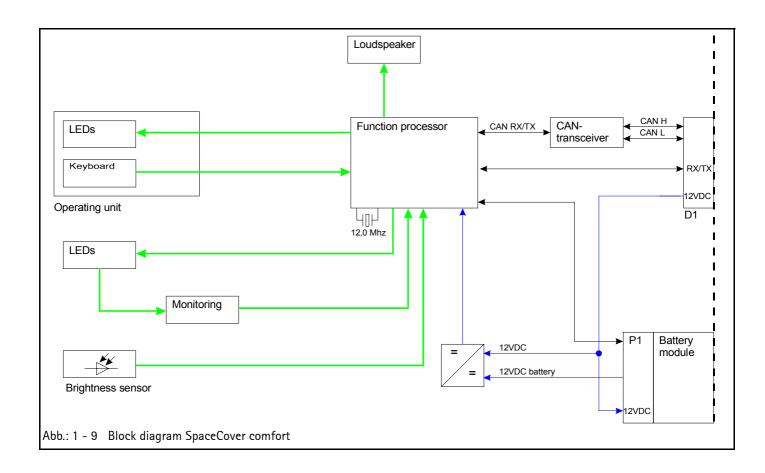

Unit Software

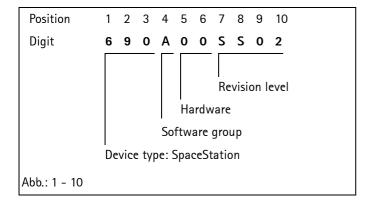

Approved Software Versions for SpaceStation

690A00SS01

- Basic software

690A00SS02

- Improved functions

Approved Software Versions for SpaceCom

695C000001 / 695C000002

- Basic software

695C010003

Adapted to hardware changes

695D010001

- Adapted to the pump software

695E010001

Adapted to the pump software

Approved Software Versions for SpaceCover comfort

692A000001

- Basic software

692A000002

Improved functions

692A000003

- Expanded functions

692C000001

- Adapted to the pump software

692E000001

Adapted to the pump software

1 - 12 SpaceStation, 2.0 gb


Software Update

The instructions for updating the software are supplied with the software itself.

CAUTION

If the device is disconnected while the software is being updated or the device or PC is switched off, a component of the software may be seriously damaged so that repairs are no longer possible. In such a case the software can no longer be updated via the PC and the device must be returned to B. Braun.

Service Program

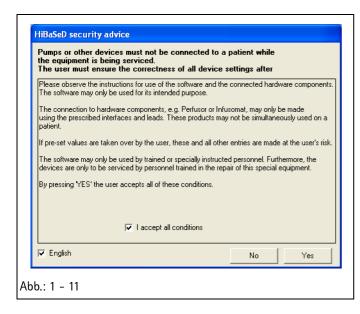

Approved Version

Note

Please note that text and / or functions of the Service Program may change depending on the software version. The following screen illustrations are only examples and represent the state when the manual was printed.

- 0.0.28.0
- 1.3.5
- 1.5.0
- 2.0.1

Connection

Before starting the Service Program the PC is to be connected to the device via the RS232 SP interface cable.

Starting the Service Program


Installation and further operation of the Service Program is described in its separate instructions for use.

- Start the "HiBaSeD.exe" program (<u>Hi</u>story, <u>Ba</u>rcode, <u>Se</u>rvice, <u>D</u>rug list) on the PC. The Service Program is loaded and started and the initial window of the Service Program is displayed.
- 2. Read the notes carefully.
- 3. Mark the field "I accept all conditions" and then the field "Yes" to confirm that you have read the notes.

Note

Click the field "English" to switch the language of the notes over to English.

1 - 14 SpaceStation, 2.0 gb

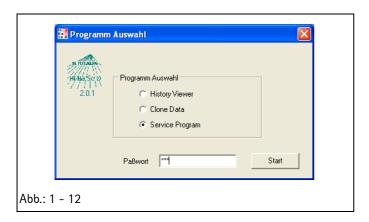




4. Enter the password and confirm it by clicking the field "Start".

The Service Program checks the PC interfaces for connected devices of the Space system. Units that were found are displayed for a short moment on the screen.

Note

At present a connected SpaceStation is not displayed.

The work window of the Service Program appears on the screen. All devices recognized are listed in the left column.

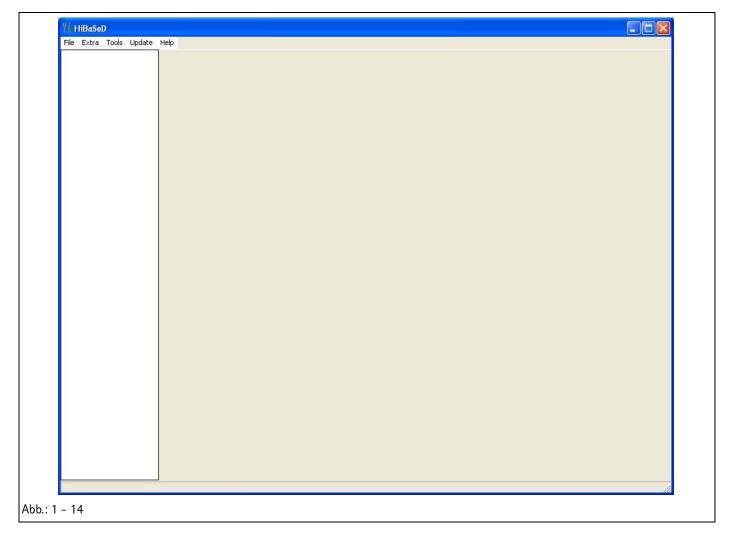




Service Program Version

- Open the "HiBaSeD" window via Help → Info The current version of the Service Program is shown in this window.
- 2. Close the window by clicking "OK".

1 - 16 SpaceStation, 2.0 gb

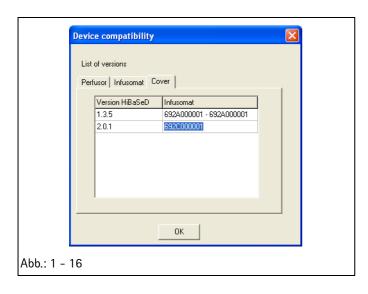

Compatibility List

- Open the "Unit Compatibility" window via Help → Decice
 compatibility. This window displays the compatibility of the
 HiBaSeD-version and the unit software version.
- 2. Close the window by clicking "OK".

Quit the Service Program

1. Exit the Service Program via *File* **⇒** *Exit*.

SpaceOnline

(only required with SpaceStation with SpaceCom)

Approved Version

Note

Please note that text and / or functions of the SpaceOnline Program may change depending on the software version. The following screen illustrations are only examples and represent the state when the manual was printed.


- 1.2.0

Connection

Before starting the SpaceOnline Program the PC is to be connected to the device via the RJ45 cross-over interface cable. In addition the PC-LAN connection must be configured with the correct IP address (e.g. 192.168.100.1). Make sure that the first three digit blocks, e.g. 192.168.100.xxx, of PC and the SpaceCom are identical

Starting the SpaceOnline Program

1. Start the Internet Browser on your PC and enter the IP-address of the SpaceCom. The SpaceOnline start screen appears.

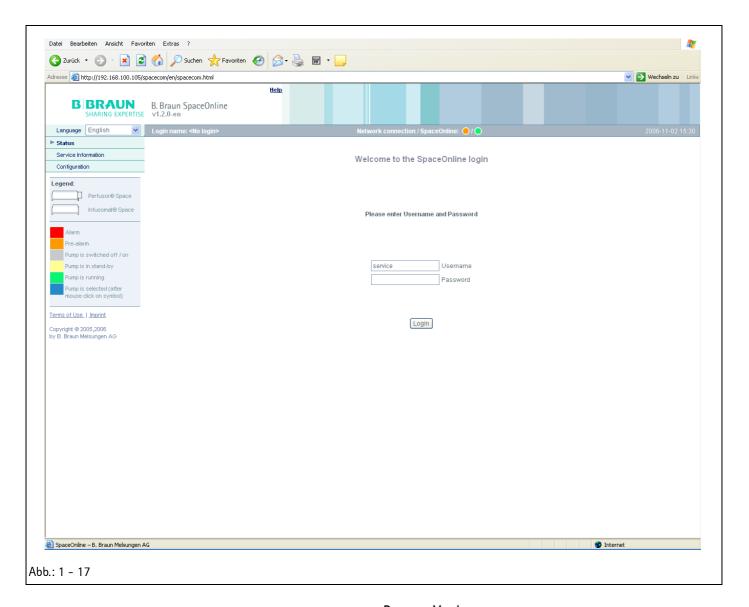

Program Version

The program version is displayed in the header of the start screen.

Operation

The instructions for use contain a more detailed description of operation.

Exit the SpaceOnline Program

1. Close the Internet Browser.

1 - 18 SpaceStation, 2.0 gb

BCCshow

Approved Version

Note

Please note that text and / or functions of the BCCshow Program may change depending on the software version. The following screen illustrations are only examples and represent the state when the manual was printed.

- 3.26

Connection

Before starting the BCCshow Program the PC is to be connected to the device via the RS232 SPCO (crossed) interface cable.


Connection Settings


- 1. Open the "BCCshow.ini" file after having installed the program.
- 2. Check the line "ComPort=X".
- 3. Substitute the X for the current ComPort number, e.g. "1".

Starting the BCCshow Program

- 1. Start the BCCshow program. The start screen appears.
- Invoke the display of the connected system via System → Start.

The SpaceStation with all pumps inserted and the set data is displayed.

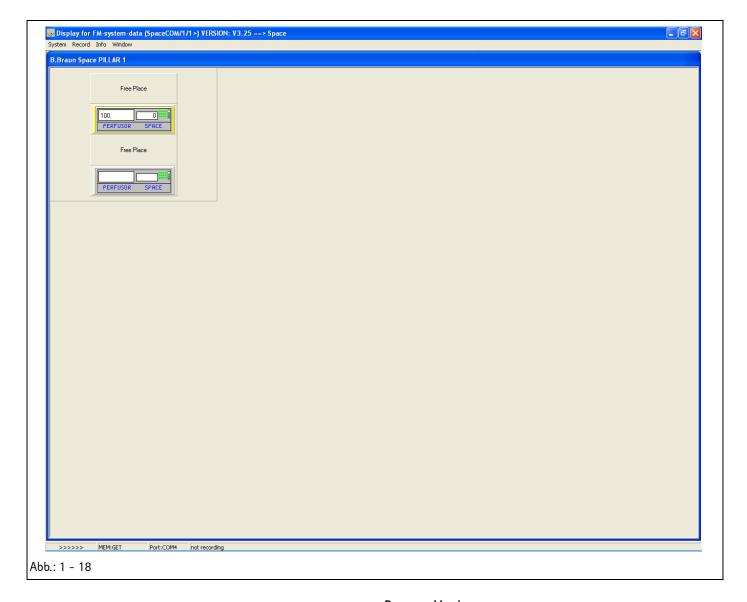

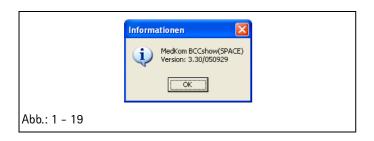

Program Version

- Open the "Information" window via *Info* → *Version*. The current program version is displayed in this window.
- 2. Close the window by clicking "OK".

1 - 20 SpaceStation, 2.0 gb

Technical Data	All technical data is indicated in the instructions for use.
Options	The functions of the individual options are detailed in the instructions for use.
	SpaceStation with SpaceCom
	Designation Part No.:
	SpaceCom retrofit kit
	The optional SpaceCom provides additional interfaces, such as Ethernet RJ45, RS232. A SpaceStation without SpaceCom can be upgraded with this retrofit kit.
Accessories	Designation Part No.
	Charger SP
	Connecting cable staff call SP 0871 3232
	Power supply cable 12 V
	Interface cable CAN SP
	Interface cable RS232 SP
	Interface cable RS232 SPCO (crossed) 0871 3237
	Converter RS232 SP 0871 3238
	W-LAN card 0871 3184
	Connecting cable SP 60cm
	Connecting cable SP 120cm 0871 3215 for connecting two columns
	PoleClamp SP
	Universal clamp

1 System Overview

For your information:	

1 - 22 SpaceStation, 1.0 gb

General

WARNING

WHILE TESTING THE UNIT AND TROUBLE SHOOTING THE OPERATOR/SERVICE TECHNICIAN MUST WORK WITH VOLTAGES UP TO 115 / 230 V AC. THESE VOLTAGES MAY CAUSE INJURIES WHICH ARE DANGEROUS TO LIFE AND LIMB. THE NATIONAL AND INTERNATIONAL SAFETY REGULATIONS ARE TO BE ADHERED TO.

Before each disassembly of a unit subsystem check the connectors, plug contacts and connections for corrosion and tight fit. These fault types are not described again in the following trouble shooting list.

The following equipment is necessary for testing the SpaceStation without SpaceCom and/or performing trouble shooting:

- PC
- Service connector SP
- Service Program HiBaSeD
- Interface cable RS232 SP
- Space system pump
- SpaceCover comfort (for unit diagnosis of the SpaceStation)

The following additional equipment is necessary for testing the SpaceStation with SpaceCom and/or performing trouble shooting:

- Interface cable CAN SP
- Interface cable RS232 SPCO (crossed)
- Interface cable RJ45 cross-over


CAUTION

Take special care when carrying out measurements on an open and switched-on unit. Short circuits and wrong measuring methods can cause serious damage to or destroy the subsystems of the device.

The unit check, calibration and trouble shooting are subdivided into numbered working steps (Unit Test Step UTS, Calibration Step CS, Trouble Shooting TS) and are based on each other.

Beginning with UTS 1 the operation described here has to be executed. The consequences of the steps performed are listed in the "Function" column. If the result corresponds to the consequence, the working step must be carried out to which reference is made in the column "If yes". If the result does not correspond with the function described, the working step in column "If no" is to be executed.

One example is given in Abb.: 2 - 1.

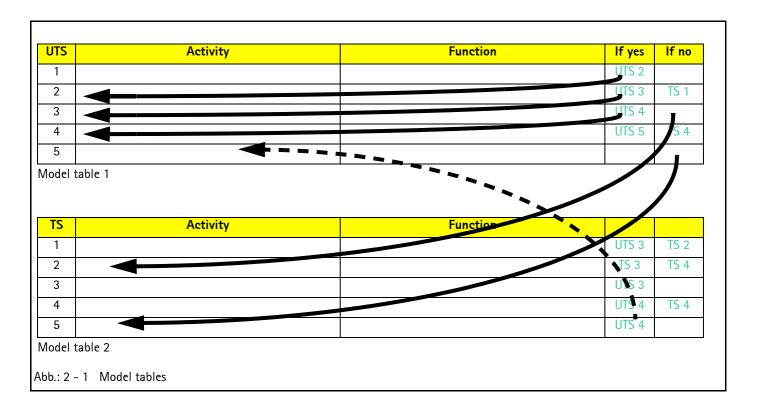

Steps for which additional information is required are described hereafter in detail.

2 - 2 SpaceStation, 1.0 gb

Alarms and Error Codes

SpaceStation without SpaceCom

Neither alarms nor error codes are actually displayed on the SpaceStation without SpaceCom.

SpaceStation with SpaceCom

	Alarm	Possible Cause	Fault Clearance
1	SpaceComAlarm indicator (red) is on	SpaceCom is still starting	
2	SpaceComAlarm indicator (red) flashes	A serious internal fault was detected in	Carry out a device check (see "Device
		the system	Check" → S. 2 - 4)

Table 2 - 1 Alarms on the SpaceCom

SpaceCover comfort

	Alarm	Possible Cause	Fault Clearance
1	One of the diodes of the status- and	A message or an alarm of a pump is dis-	
	alarm display line is on or flashes	played	
2	The status indicator (on the right-hand	System not properly configured	Check configuration
	side of the On-/Off-button) flashes red		
3	The status indicator (on the right-hand	A serious internal fault was detected in	Carry out a device check (see "Device
	side of the On-/Off-button) lights up red	the system	Check" → S. 2 - 4)
4	The first LED in the top charge condition	Battery module of the Space Cover com-	Connect device to voltage supply
	display of the operator and status display	fort nearly discharged (type: pre-alarm)	
	field of the SpaceCover comfort flashes		
	yellow		
5	The first LED in the bottom charge condi-	Battery module of theSpaceCom nearly	Connect device to voltage supply
	tion display of the operator and status	discharged (type: pre-alarm)	
	display field of the SpaceCover comfort		
	flashes yellow		
6	The first LED in the top charge condition	Battery module of the SpaceCover com-	Connect device to voltage supply
	display of the operator and status display	fort discharged	
	field of the SpaceCover comfort flashes		
	red		
7	The first LED in the bottom charge condi-	Battery module of the SpaceCom dis-	Connect device to voltage supply
	tion display of the operator and status	charged	
	display field of the SpaceCover comfort		
	flashes red		

Table 2 - 2 Alarms of the SpaceCover comfort (Part 1 of 2)

SpaceStation, 2.0 gb 2 - 3

	Alarm	Possible Cause	Fault Clearance
8	The first LED in the top charge condition	Battery module of the SpaceCover com-	Replace battery module
	display of the operator and status display	fort defective or too old	
	field of the SpaceCover comfort lights up		
	red		
9	The first LED in the bottom charge condi-	Battery module of the SpaceCom defec-	Replace battery module
	tion display of the operator and status	tive or too old	
	display field of the SpaceCover comfort		
	lights up red		

Table 2 - 2 Alarms of the SpaceCover comfort (Part 2 of 2)

The Most Important Error Modes

The following list specifies the most important error modes and their clearance.

Note

The error list is in preparation.

Device Check

UTS	Activity	Function	If yes	If no
1	Detach or loosen all connections and devices from		UTS 2	
	the SpaceStation			
2	Connect the SpaceStation via the mains cable to		UTS 3	
	the mains			
3	Insert one pump after the other in each slot	The pump is locked	UTS 4	TS 1
4		The pump can be released at any slot	UTS 5	TS 1
5	Plug service connector SP on connector "F3"		UTS 6	
6	Insert the pump in the topmost slot and trigger a	The red LED of the service connector SP lights up	UTS 7	TS 4
	staff call			
7	Remove the pump while it is switched on	The red LED of the service connector SP goes out	UTS 8	TS 4
8	Insert the switched-on pump in each other slot	The red LED of the service connector SP lights up	UTS 9	TS 4
		when the lock engages		
9	A SpaceCover comfort with SpaceStation is in-		UTS 10	UTS 20
	stalled			
10	Mount and lock the SpaceCover comfort on the	All luminous fields of the status and alarm display	UTS 11	TS 5
	SpaceStation	line light up briefly during the self-test		
11		A short audible alarm sounds	UTS 12	TS 7

Table 2 - 3 Device check (Part 1 of 2)

2 - 4 SpaceStation, 2.0 gb

UTS	Activity	Function	If yes	If no
12		All LEDs of the operator and status display field light	UTS 13	TS 5
		up briefly		
13		The green LED on the operator and status display	UTS 14	TS 5
		field lights up		
14		When the battery module is inserted the charge	UTS 15	TS 9
		condition is displayed in the top charge status dis-		
		play of the operator and status display field		
15	Change loudness of the alarm tone	The loudness changes	UTS 16	TS 11
16	Insert pump in the topmost slot and start an infu-	The infusion is displayed green in the status and	UTS 17	TS 13
	sion	alarm display line		
17	Insert the pump in each other slot and start an in-	The infusion is displayed green in the status and	UTS 18	TS 14
	fusion	alarm display line		
18	Trigger an alarm on the pump	The alarm is displayed red (orange in case of a pre-	UTS 19	TS 15
		alarm) in the status and alarm display line		
19	If a SpaceCom is integrated in the SpaceStation			UTS 20
	carry out the device check according to Chapter			
	"Device Check SpaceCom" (➡ S. 2 – 5).			
20	Detach or loosen all connections and devices from		This step t	erminates
	the SpaceStation		the devi	ce check

Table 2 - 3 Device check (Part 2 of 2)

Device Check SpaceCom

GS-	Activity	Function	If yes	If no
COM				
1	Disconnect the SpaceStation from the mains and		UTS-COM 2	
	switch off a SpaceCover comfort which may be			
	fitted			
2		A battery module is installed in the SpaceStation	UTS-COM 3	UTS-COM 4
3	Insert a battery module in the SpaceStation		UTS-COM 4	
4	Connect the SpaceStation to the mains	Fan in the housing back panel starts running for	UTS-COM 5	TS 17
		appr. 1 sec.		
5		The status indicator (green) of the SpaceCom	UTS-COM 6	TS 20
		lights up		
6		The alarm indicator (red) of the SpaceCom lights	UTS-COM 7	TS 20
		up for approx. 1 minute and goes out		

Table 2 - 4 Device check SpaceCom (Part 1 of 2)

SpaceStation, 2.0 gb 2 - 5

GS-	Activity	Function	If yes	lf no	
СОМ					
7		The charge condition of the battery module is dis-	UTS-COM 8	TS 22	
		played in the bottom charge condition display of			
		the operator and status display field of the Space-			
		Cover comfort			
8	Connect the SpaceStation with the interface ca-		UTS-COM 9		
	ble RJ45 cross-over via the Ethernet (RJ45) con-				
	nection of the SpaceCom to a PC				
9	Start the SpaceOnline Program (see "SpaceOnli-	The data of the SpaceStation and the pumps in-	UTS-COM	TS 20	
	ne" ⇒ S. 1 – 17)	serted is displayed on the PC monitor	10		
10	Connect the SpaceStation with the interface ca-		UTS-COM		
	ble RS232 SPCO (crossed) via the RS 232 connec-		11		
	tor of the SpaceCom to a PC				
11	Start the BCCshow Program (see "BCCshow" ▶	The data of the SpaceStation and the pumps in-	UTS-COM	TS 20	
	S. 1 – 19).	serted is displayed on the PC monitor	12		
12	Detach or loosen all connections and devices		This step terr	minates the	
	from the SpaceStation	dev		vice check	

Table 2 - 4 Device check SpaceCom (Part 2 of 2)

Trouble Shooting

Note

The following trouble shooting cannot be carried out independently. It is based on the precise observance of the steps for the device check (see "Device Check" > S. 2 - 4). From there reference is made to the corresponding trouble shooting steps.

TS	Activity	Function	If yes	If no
1	Replace the module lock		UTS 3	
2	Check the voltage 12 V (-0.6 V / $+3.5 \text{ V}$) DC at the interface board plug contacts (connection power supply – interface board)	_	TS 4	TS 3
3	Replace power supply		UTS 4	
4	Replace interface board		UTS 4	

Table 2 - 5 Trouble shooting (Part 1 of 2)

2 - 6 SpaceStation, 2.0 gb

TS	Activity	Function	If yes	lf no
5	Exchange processor PCB SPCC	All luminous fields of the status and alarm display line light up briefly during the self-test	UTS 10	TS 6
		All LEDs of the operator and status display field light up briefly		
		The green LED on the operator and status display field lights up		
6	Replace interface board in the SpaceStation		UTS 10	
7	Replace loudspeaker	A short audible alarm sounds	UTS 14	TS 8
8	Exchange processor PCB SPCC		UTS 14	
9	Replace interface board in the SpaceStation	The charge state of the battery module is displayed on the operator and status display field	UTS 15	TS 10
10	Exchange the housing upper part SPCC		UTS 15	
11	Exchange the housing upper part SPCC	The loudness changes	UTS 16	TS 12
12	Exchange processor PCB SPCC		UTS 16	
13	Exchange processor PCB SPCC	The infusion is displayed green in the status and alarm display line	UTS 18	TS 14
14	Replace interface board in the SpaceStation		UTS 18	
15	Exchange processor PCB SPCC	The alarm is displayed red in the status and alarm display line	UTS 20	TS 16
16	Replace interface board in the SpaceStation		UTS 20	
17	Pull off the fan connecting cable on the SPCO board and trigger the fan with 12 V DC. Connect the (+) pole to the connector pole with red wires and the (-) pole to the connector pole with black wires	The fan is running	TS 19	TS 18
18	Exchange fan		UTS-COM 5	
19		The status indicator (green) of the SpaceCom lights up	TS 20	TS 21
20	Replace the SPCO board		UTS-COM 5	
21	Replace interface board in the SpaceStation		UTS-COM 5	
22	Replace battery module of the SpaceStation	The charge condition of the battery module is displayed in the bottom charge condition display of the operator and status display field of the Space-Cover comfort	UTS-COM 8	TS 23
23	Replace the SPCO board	The charge condition of the battery module is displayed in the bottom charge condition display of the operator and status display field of the Space-Cover comfort	UTS-COM 8	TS 24
24	Replace interface board in the SpaceStation		UTS-COM 8	

Table 2 - 5 Trouble shooting (Part 2 of 2)

SpaceStation, 2.0 gb 2 - 7

2

Unit Diagnosis / Calibration

For your information:	

2 - 8 SpaceStation, 2.0 gb

3.1 General on the SpaceStation without Space-

Remarks on Disassembly / Assembly of the SpaceStation without SpaceCom

WARNING

DURING DISASSEMBLY AND ASSEMBLY THE OPERATOR/SERVICE TECHNICIAN MUST WORK WITH VOLTAGES UP TO 115 / 230 V AC. THESE VOLTAGES MAY CAUSE INJURIES WHICH ARE DANGEROUS TO LIFE AND LIMB. THE NATIONAL AND INTERNATIONAL SAFETY REGULATIONS ARE TO BE ADHERED TO. THE MAINS CABLE IS TO BE REMOVED.

Before disassembling the unit, the system must be checked (see "Device Check" → S. 2 - 4) to isolate the part to be exchanged.

All necessary steps to disassemble or dismount the complete unit with all its subsystems and spare parts are detailed in the following description. Steps that are not necessary can be skipped.

Note

Special screws for plastic housings are used in this unit. Pay attention to the corresponding notes when you fit the screws.

SpaceStation, 2.0 gb 3 - 1

Service Parts and Screw Kit

All small parts, such as cover caps, are contained in a SpaceStation service part kit.

Designation Ord. No.

with:

Release button

Release seal

Seal M-2K

Cover cap for housing

Cover cap for handle

Housing foot SPS

Seal, mains F1A

Pole clamp lever

Pole clamp spring

0-ring 6.07 x 1.78

0-ring 12.0 x 2.0

All screws used in the device are included in a SpaceStation screw kit.

Designation Ord. No.

Screw kit SpaceStation 3477 4343

Screw EJOT 30x9 WN 5452 TORX 10IP A2

Screw EJOT 30x12 WN 5452 TORX 10IP A2

Screw EJOT 30x16 WN 5451 TORX 10IP A2

Screw M5x45 A2 DIN912 TORX 25

Countersunk screw M5x12 TORX 25

Fillister-head screw M5x12 EN-ISO14583 A2

Nut M4

Insulating washer M3

Washer M4


Washer M5

Serrated lock washer M4

Serrated lock washer M5

3 - 2 SpaceStation, 2.0 gb

3.2 Tube Guide

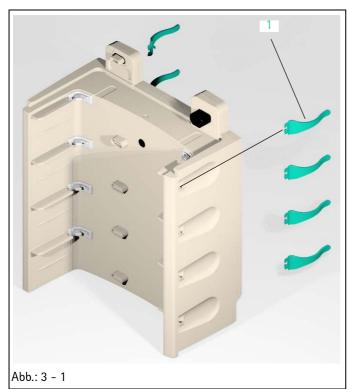

Legende zu Abb. 3 - 1: ItemDesignation

1 Tube guide

Designation Ord. No.

Disassembly


1. Press the rotary knob of the tube guide out and pull the tube guide (Abb.: 3 - 1 / Item 1) out of the housing.

Note

When the tube guide is broken, remove the remaining parts from the housing.

SpaceStation, 1.0 gb

3.3 Pole Clamp Guide

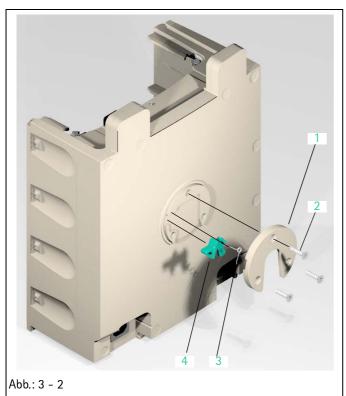

Legende zu Abb. 3 - 2: ItemDesignation


- 1 Pole clamp guide
- 2 Countersunk screw M5x12 TORX 25
- 3 Pole clamp spring
- 4 Pole clamp lever

Disassembly

- 1. Loosen three screws (Abb.: 3 2 / Item 2) and remove the screws together with the pole clamp guide (Abb.: 3 2 / Item 1).
- 2. Remove pole clamp lever (Abb.: 3 2 / Item 4) and pole clamp spring (Abb.: 3 2 / Item 3).

3 - 4 SpaceStation, 1.0 gb

3.4 Housing Back Panel

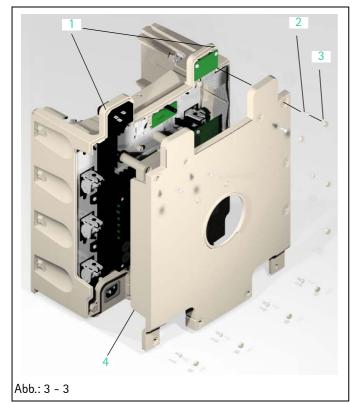

Legende zu Abb. 3 - 3:

Item Designation

- 1 Housing notch
- 2 Screw EJOT 30x9 WN 5452 TORX 10IP A2
- 3 Cover cap
- 4 Housing back panel

Designation Ord. No.

Housing back panel SPS 3452 1089

Cover cap and screws

(see "Service Parts and Screw Kit" → S. 3 – 2)

Disassembly


- 1. Pierce ten cover caps (Abb.: 3 3 / Item 3) with a screwdriver and pull caps out.
- 2. Loosen nine screws (Abb.: 3 3 / Item 2) and remove screws carefully together with the back panel (Abb.: 3 3 / Item 4) from the housing.

Note

Lift the housing back panel carefully at the bottom over the seat of the pole clamp guide and push the back panel over the housing notch (Abb.: 3 - 3 / Item 1).

SpaceStation, 1.0 gb 3 - 5

3.5 Device Bracket

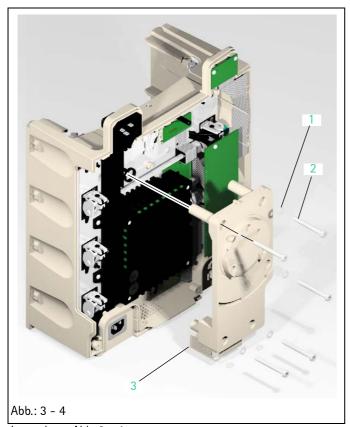

Legende zu Abb. 3 - 4:

ItemDesignation

- 1 Serrated lock washer M5
- 2 Screw M5x45 A2 DIN912 TORX 25
- 3 Device bracket

Disassembly

1. Loosen five screws (Abb.: 3 - 4 / Item 2) and remove screws together with the serrated lock washers (Abb.: 3 - 4 / Item 1) and the device bracket (Abb.: 3 - 4 / Item 3).

3 - 6 SpaceStation, 2.0 gb

3.6 Power Supply

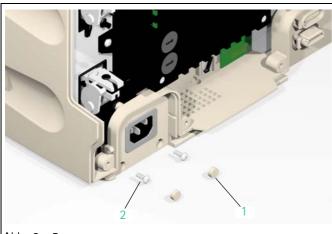


Abb.: 3 - 5

Legende zu Abb. 3 - 5:

ItemDesignation

- 1 Cover cap
- 2 Screw EJOT 30x9 WN 5452 TORX 10IP A2



Cover cap, screws, washer,

serrated lock washers and nuts


(see "Service Parts and Screw Kit" → S. 3 - 2)

Mains seal, bottom

(see "Service Parts and Screw Kit" → S. 3 - 2)

Disassembly

- 1. Pierce two cover caps (Abb.: 3 5 / Item 1) with a screwdriver and pull caps out.
- 2. Unscrew two screws (Abb.: 3 5 / Item 2) and remove them from the mains connector holder.

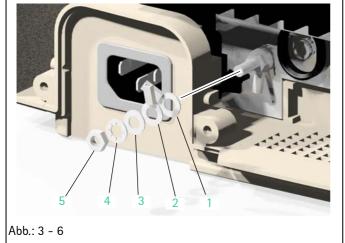

Legende zu Abb. 3 - 6:

ItemDesignation


- 1 Cable lug for connection to the power supply
- 2 Cable lug for connection to connector "F1B"
- 3 Washer M4
- 4 Serrated lock washer M4
- 5 Nut M4

Note

Note the position of the protective conductor cable on the grounding bolt.

3. Loosen the nut (Abb.: 3 - 6 / Item 5) and remove nut together with the serrated lock washer (Abb.: 3 - 6 / Item 4), washer (Abb.: 3 - 6 / Item 3), the protective conductor cable to the power supply (Abb.: 3 - 6 / Item 1) and the protective conductor cable to connector "F1B" (Abb.: 3 - 6 / Item 2) from the grounding bolt.

SpaceStation, 2.0 gb 3 - 7

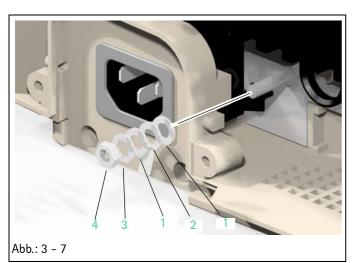

Legende zu Abb. 3 - 7:

Item Designation

- 1 Washer M4
- 2 Cable lug for connection to connector "F1A"
- 3 Serrated lock washer M4
- 4 Nut M4
- Abb.: 3 8

Legende zu Abb. 3 – 8: ItemDesignation

Screw EJOT 30x9 WN 5452 TORX 10IP A2


- 4. Loosen the nut (Abb.: 3 7 / Item 4) and remove nut together with the serrated lock washer (Abb.: 3 7 / Item 3), washers (Abb.: 3 7 / Item 1) and the protective conductor cable to connector "F1A" (Abb.: 3 7 / Item 2) from the grounding bolt.
- 5. Carefully pull off the connector of the power supply wires from the interface board.

Note

Note or mark the polarity.

6. Loosen two screws (Abb.: 3 - 8 / Item 1) and take them out of the power supply housing (connector).

3 - 8 SpaceStation, 2.0 gb

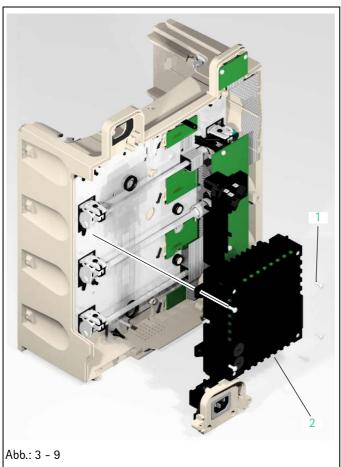


Legende zu Abb. 3 – 9: ItemDesignation

- 1 Screw EJOT 30x12 WN 5451 TORX 10IP A2
- 2 Power supply

7. Loosen four screws (Abb.: 3 - 9 / Item 1) and take the power supply (Abb.: 3 - 9 / Item 2) with the mains connector holder out of the housing.

SpaceStation, 1.0 gb

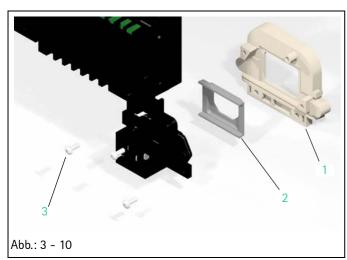

Legende zu Abb. 3 - 10:

ItemDesignation


- 1 Mains connector holder
- 2 Mains seal F1A
- 3 Screw EJOT 30x9 WN 5452 TORX 10IP A2

Disassembly

1. Loosen three screws (Abb.: 3 - 10 / Item 3) and remove screws together with the mains connector holder (Abb.: 3 - 10 / Item 1) and the mains seal F1A (Abb.: 3 - 10 / Item 2) from the power supply.

3 - 10 SpaceStation, 1.0 gb

3.7 Release Button

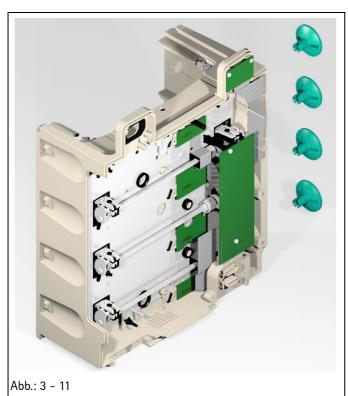

Legende zu Abb. 3 - 11: ItemDesignation

1 Release button

Designation

Ord. No.

Release button and release seal

(see "Service Parts and Screw Kit" ⇒ S. 3 – 2)

Disassembly


1. Press four release buttons (Abb.: 3 - 11 / Item 1) carefully from the inside from the locking shaft using a screwdriver and pull the buttons out of the housing.

Note

Check the release buttons for damage, especially within the notch area, before fitting the buttons.

SpaceStation, 1.0 gb

3.8 Interface Board / Module Lock

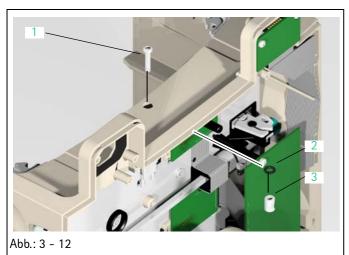

Legende zu Abb. 3 - 12: ItemDesignation


- 1 Module lock screw
- 2 O-Ring 6.07 x x1.78
- 3 Module lock sleeve

Designation	Ord. No.
Interface board SPS	3452 1100
Connector holder, data, SPS	3452 1160
Module lock SPS	3452 1119
Cover cap, screws, insulating washers, 0-rings (see "Service Parts and Screw Kit" → S. 3 - 2)	

Disassembly

- 1. Move the lock towards the SpaceCover or another SpaceStation in the closed position.
- 2. Loosen one screw (Abb.: 3 12 / Item 1) of the module lock and remove it out of the housing.
- 3. Pull the module lock sleeve (Abb.: 3 12 / Item 3) out of the housing.
- 4. Check the O-ring (Abb.: 3 12 / Item 2) for damage, and remove it out of the housing if necessary.

3 - 12 SpaceStation, 1.0 gb

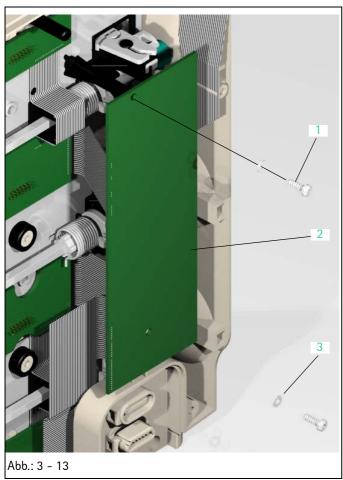

Legende zu Abb. 3 - 13:

ItemDesignation

- 1 Screw EJOT 30x9 WN 5452 TORX 10IP A2
- 2 Interface board
- 3 Insulating washer M3

- 5. Loosen two screws (Abb.: 3 13 / Item 1) and remove screws together with the insulating washers (Abb.: 3 13 / Item 3) from the interface board (Abb.: 3 13 / Item 2).
- 6. Put the interface board carefully aside for all further activities.

Note

Take care not to squeeze the interface board ribbon cable.

SpaceStation, 1.0 gb 3 - 13



Legende zu Abb. 3 - 14: ItemDesignation

- 1 Release button
- 2 Seal M-2K


7. Release all rotary knobs (Abb.: 3 – 14 / Item 1) of the module lock.

Note

Put a release button on the release shaft without engaging it to unlock the rotary knobs. To lock the rotary knobs press them simultaneously in by hand per slot.


8. Press 8 seals M-2K (Abb.: 3 - 14 / Item 2) carefully out of the housing, press the rotary knobs in the locking position and pull the seals M-2K off the rotary knobs.

3 - 14 SpaceStation, 1.0 gb

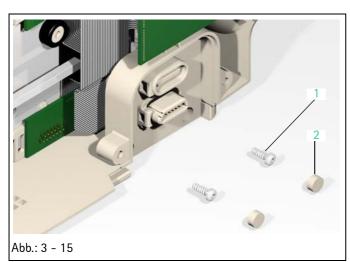

Legende zu Abb. 3 - 15: ItemDesignation

- 1 Screw EJOT 30x9 WN 5452 TORX 10IP A2
- 2 Cover cap

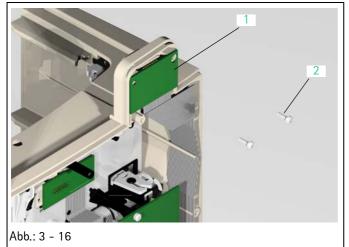

Legende zu Abb. 3 - 16:

Item Designation

- 1 Connector board F5
- 2 Screw EJOT 30x9 WN 5452 TORX 10IP A2


- 9. Pierce two cover caps (Abb.: 3 15 / Item 2) with a screw-driver and pull caps out.
- 10. Loosen two screws (Abb.: 3 15 / Item 1) and remove them out of the data connector holder.

- 11. Loosen two screws (Abb.: 3 16 / Item 2) and remove them from the connector board F5 (Abb.: 3 16 / Item 1).
- 12. Press the connector board F5 carefully out of the housing.

Note

The connector board F5 is integrated in the interface board.

SpaceStation, 1.0 gb 3 - 15

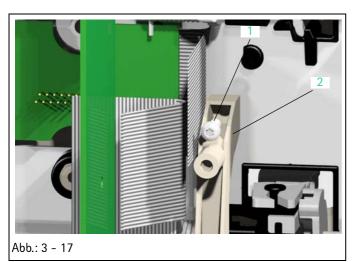


Legende zu Abb. 3 - 17:

Item Designation

- 1 Screw EJOT 30x9 WN 5452 TORX 10IP A2
- 2 Data connector holder

13. Loosen one screw (Abb.: 3 - 17 / Item 1) and remove it out of the data connector holder (Abb.: 3 - 17 / Item 2).

3 - 16 SpaceStation, 1.0 gb

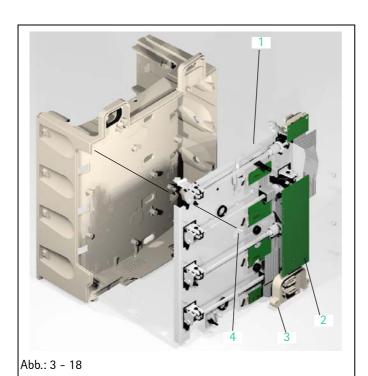

Legende zu Abb. 3 - 18:

Item Designation


- 1 Module lock
- 2 Interface board
- 3 Data connector holder
- 4 Screw EJOT 30x9 WN 5452 TORX 10IP A2

14. Loosen three screws (Abb.: 3 - 18 / Item 4) and remove screws together with the module lock (Abb.: 3 - 18 / Item 1), the interface board (Abb.: 3 - 18 / Item 2) and the data connector holder (Abb.: 3 - 18 / Item 3) out of the housing.

Note

Note the ribbon cable length to the interface board when dismounting the data connector holder. The ribbon cable must not be kinked.


SpaceStation, 1.0 gb 3 - 17

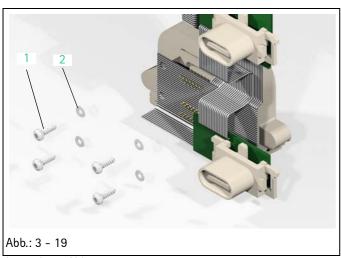

Legende zu Abb. 3 - 19:

ItemDesignation

- 1 Screw EJOT 30x9 WN 5452 TORX 10IP A2S
- 2 Insulating washer M3

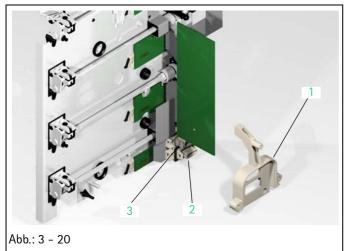

Legende zu Abb. 3 - 20:

Item Designation

- 1 Data connector holder
- 2 Connector F3
- 3 Connector F4

Disassembly


1. Loosen fours screws (Abb.: 3 - 19 / Item 1) and remove screws together with the insulating washers (Abb.: 3 - 19 / Item 2).

2. Press connectors F3 (Abb.: 3 - 20 / Item 2) and F4 (Abb.: 3 - 20 / Item 3) carefully out of the data connector holder (Abb.: 3 - 20 / Item 1).

Note

The connectors F3 and F4 are integrated in the interface board.

3 - 18 SpaceStation, 1.0 gb

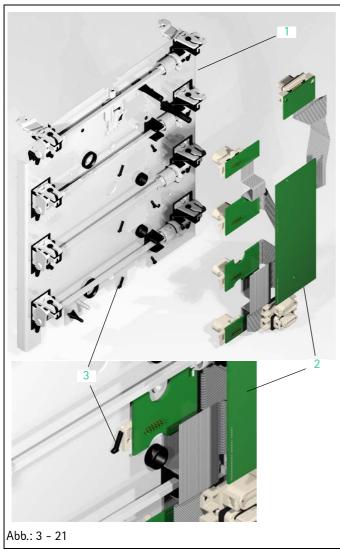

Legende zu Abb. 3 - 21: ItemDesignation

- 1 Module lock
- 2 Connector board
- 3 Connector board lockings

3. Press the connector board lockings (Abb.: 3 - 21 / Item 3) F2A, F2B, F2C, and F2D carefully aside, push the connectors downwards, pull them out of the module lock (Abb.: 3 - 21 / Item 1) and remove them with the interface board (Abb.: 3 - 21 / Item 2) from the module lock.

Note

The connectors F2A, F2B, F2C, and F2D are connected to each other and integrated in the interface board.

SpaceStation, 1.0 gb

3.9 Housing

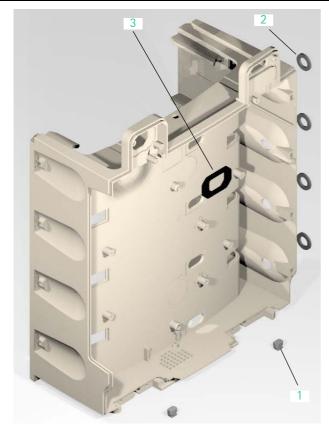


Abb.: 3 - 22

Legende zu Abb. 3 - 22: ItemDesignation

- 1 Housing foot SPS
- 2 Release seal
- 3 Mains sealF1B

Designation Ord. No.

Screws, seals and housing foot SPS

(see "Service Parts and Screw Kit" ⇒ S. 3 – 2)

Disassembly

- 1. Pull two housing feet (Abb.: 3 22 / Item 1) out of the housing.
- 2. Remove four release seals (Abb.: 3 22 / Item 2) from the housing.

Note

The seals are self-adhesive. Remove any adhesive residues after disassembly.

3. Remove mains seal F1B (Abb.: 3 - 22 / Item 3) out of the housing.

Note

The housing must not be disassembled further. The seals behind the corner profiles on the right and left side can only be inserted correctly by means of special tools.

3 - 20 SpaceStation, 1.0 gb

3.10 Assembly / Installation

Assembly or installation of the modules and subsystems is done in reverse order of disassembly. Special steps to be observed are described hereafter in detail.

Only new cover caps are to be used.

Special Screws

Special screws for plastic housings are used in this unit. The screws are not self-cutting but produce a thread in the plastic of the housing through deformation when fitted in for the first time.

If the beginning of the thread is not engaged when the screw is fitted, a new thread is produced and the old thread is destroyed so that the security of the fixing can no longer be guaranteed.

Proceed as follows to fit the special screws:

- 1. Put the screw on the thread.
- Rotate screw anti-clockwise (loosen) until a faint click can be heard. This click is produced when the screw thread drops into the existing thread.
- 3. Screw in the screw and tighten with the defined torque.

Interface Board / Module Lock

Note

Before the interface board is installed make sure that the two housing feet are inserted in the housing.

Note

When installing the interface board observe the mounting sequence, i.e. first attach the interface board to the module lock and then install the complete group in the housing.

1. Tighten the connector board screws with a torque of 0.25 to 0.05 Nm.

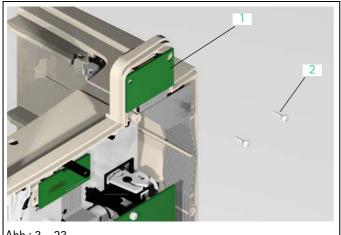



Abb.: 3 - 23

Legende zu Abb. 3 - 23:

ItemDesignation


- 1 Connector board F5
- 2 Screw EJOT 30x9 WN 5452 TORX 10IP A2

SpaceStation, 1.0 gb 3 - 21

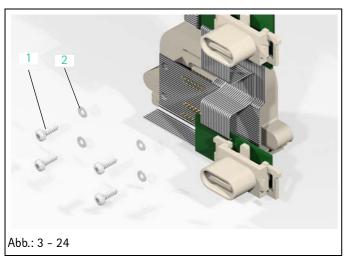

Legende zu Abb. 3 - 24: ItemDesignation

- 1 Screw EJOT 30x9 WN 5452 TORX 10IP A2S
- 2 Insulating washer M3

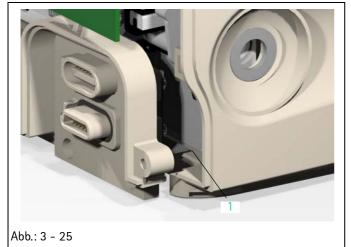

Legende zu Abb. 3 - 25: ItemDesignation

1 Rubber cable


2. Tighten the data connector holder screws to connectors F3 and F4 with a torque of 0.25 – 0.05 Nm.

3. Install the subsystem interface board / module lock in the housing in such a way that the rubber cable lies in the housing bottom.

CAUTION

When mounting the data connector holder make sure that the rubber cable is not squeezed between the housing and the connector holder. This would result in an incorrect sealing of the housing and damage to the rubber cable.

3 - 22 SpaceStation, 1.0 gb

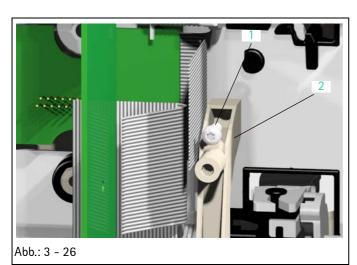

Legende zu Abb. 3 - 26: ItemDesignation

- 1 Screw EJOT 30x9 WN 5452 TORX 10IP A2
- 2 Data connector holder


- 4. Tighten the data connector holder screw with a torque of 0.25 0.05 Nm.
- 5. Before fitting the M-2K seals check them for damage and replace if necessary.

Release Button

Note

Check the release buttons for damage, especially within the notch area, before fitting the buttons.

SpaceStation, 1.0 gb 3 - 23

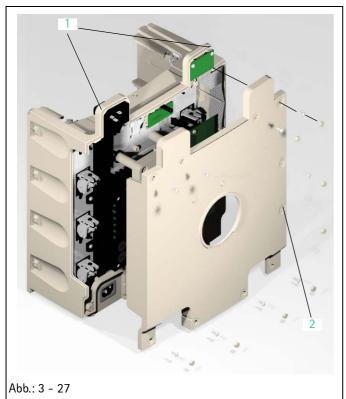

Legende zu Abb. 3 - 27: ItemDesignation

- 1 Housing notch
- 2 Empty mounting hole

Housing Back Panel

- Insert the housing back panel on the notches at the two connector coverings into the housing and rotate the back panel in its final position.
- 2. No screw is inserted in the third mounting hole (from the top) on the right-hand side of the back panel. The hole must be covered with a cover cap.

3.11 Checks after Repair

- Check the device to ensure safe functionality of the unit (see "Device Check"

 S. 2 - 4).
- Depending on the work carried out the specific steps of the TSC must be performed (see "Technical Safety Check (TSC)" ➡ S. 7 - 1).
- 3. The activities carried out must be recorded in the following check list.

3 - 24 SpaceStation, 1.0 gb

Check List for Checks after Repair

Visual Inspection		Electrical Safety		Functional Inspection
	acc	ording to IEC / EN 60601-1		
	or \	/DE 0750 and VDE 0751		
Cleanliness		Mains voltage according to the TSC		Locking with second SPS
Completeness		Protective conductor resistance ac-		Pump locking of all slots
Damage and faults affecting safety		cording to the TSC		
Damage to and readability of the label		Earth leakage current according to the	Cor	nnect unit to mains.
Cover caps		TSC	Ger	neral functional check with SpaceCover
Connectors			con	nfort and pump of every slot.
Well running and sealing of the module				Voltage supply of all slots
lock				Alarm transmission
Mains cable				Staff call transmission
Accessories				

SpaceStation, 1.0 gb

3

Disassembly / Assembly SpaceStation

For your information:	

3 - 26 SpaceStation, 1.0 gb

4.1 General on the SpaceStation with SpaceCom

Remarks on Disassembly / Assembly of the SpaceStation with SpaceCom

WARNING

DURING DISASSEMBLY AND ASSEMBLY THE OPERATOR/SERVICE TECHNICIAN MUST WORK WITH VOLTAGES UP TO 115 / 230 V AC. THESE VOLTAGES MAY CAUSE INJURIES WHICH ARE DANGEROUS TO LIFE AND LIMB. THE NATIONAL AND INTERNATIONAL SAFETY REGULATIONS ARE TO BE ADHERED TO. THE MAINS CABLE IS TO BE REMOVED.

Before disassembling the unit, the system must be checked (see "Device Check" → S. 2 - 4) to isolate the part to be exchanged.

All necessary steps to disassemble or dismount the complete unit with all its subsystems and spare parts of the SpaceCom are detailed in the following description. All further disassembly- and assembly steps correspond to the SpaceStation without SpaceCom and are decribed in Chapter "Disassembly / Assembly SpaceStation" (S. 3 - 1). Steps that are not necessary can be skipped.

Note

Special screws for plastic housings are used in this unit. Pay attention to the corresponding notes when you fit the screws.

SpaceStation, 2.0 gb 4 - 1

Disassembly / Assembly SpaceCom

Service Parts and Screw Kit

All small parts, such as cover caps, are contained in a SpaceStation service part kit.

Designation Ord. No.

Service part kit SpaceStation 3477 4335

with:

Release button

Release seal

Seal M-2K

Cover cap for housing

Cover cap for handle

Housing foot SPS

Mains seal F1A

Pole clamp lever

Pole clamp spring

0-ring 6.07 x 1.78

0-ring 12.0 x 2.0

All screws used in the device are included in a SpaceStation screw kit.

Designation Ord. No.

Screw kit SpaceStation 3477 4343

Screw EJOT 30x9 WN 5452 TORX 10IP A2

Screw EJOT 30x12 WN 5452 TORX 10IP A2

Screw EJOT 30x16 WN 5451 TORX 10IP A2

Screw M5x45 A2 DIN912 TORX 25

Countersunk screw M5x12 TORX 25

Fillister-head screw M5x12 EN-ISO14583 A2

Nut M4

Insulating washer M3

Washer M4


Washer M5

Serrated lock washer M4

Serrated lock washer M5

4 - 2 SpaceStation, 2.0 gb

4.2 Battery Module

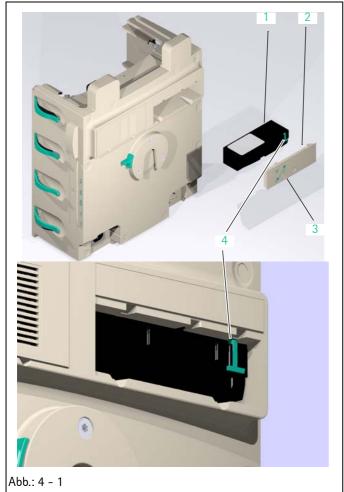

Legende zu Abb. 4 - 1: ItemDesignation


- 1 Battery module
- 2 Battery compartment cover
- 3 Battery compartment cover lock
- 4 Battery module lock

Designation	Ord. No.
Battery pack SP (NIMH)	0871 3180
Battery compartment cover SPC	3452 1232
Disassembly	

- 1. Rotate the lock (Abb.: 4 1 / Item 3) on the battery compartment cover (Abb.: 4 1 / Item 2) and remove the battery compartment cover.
- 2. Push the lock (Abb.: 4 1 / Item 4) on the battery module downward and remove the battery module (Abb.: 4 1 / Item 1) out of the battery compartment.

SpaceStation, 2.0 gb 4 - 3

4.3 W-LAN Module

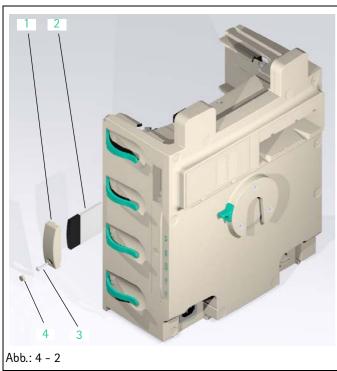

Legende zu Abb. 4 - 2:

Item Designation

- 1 W-LAN cover
- 2 W-LAN module
- 3 Screw EJOT 30x12 WN 5452 TORX 10IP A2
- 4 Cover cap

Designation	Ord. No.
W_LAN module	on request
W-LAN cover SPCO	3452 1049
W-LAN cover SPCO	3452 1050

Cover cap and screw


(see "Service Parts and Screw Kit" → S. 4 - 2)

Disassembly

- 1. Pierce cover cap (Abb.: 4 2 / Item 4) with a screwdriver and pull cap out.
- 2. Unscrew screw (Abb.: 4 2 / Item 3) and remove screw together with the W-LAN cover (Abb.: 4 2 / Item 1).
- 3. Pull W-LAN module (Abb.: 4 2 / Item 2) out of the holder.

4 - 4 SpaceStation, 2.0 gb

4.4 Tube Guide

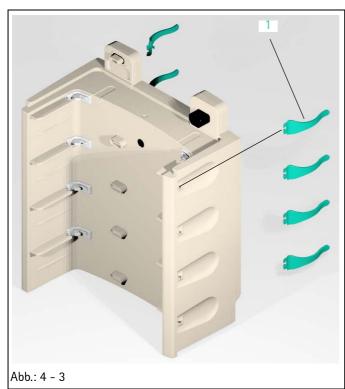

Legende zu Abb. 4 - 3: ItemDesignation

1 Tube guide

Designation Ord. No.

Disassembly


1. Press the rotary knob of the tube guide out and pull the tube guide (Abb.: 4 - 3 / Item 1) out of the housing.

Note

When the tube guide is broken, remove the remaining parts from the housing.

SpaceStation, 2.0 gb 4 - 5

4.5 Pole Clamp Guide

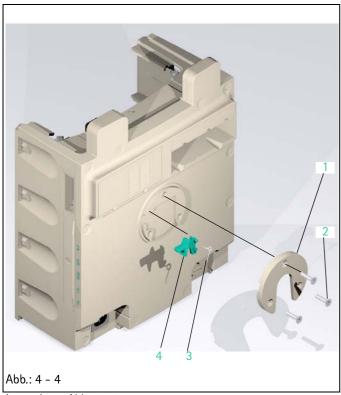

Legende zu Abb. 4 – 4: ItemDesignation

- 1 Pole clamp guide
- 2 Countersunk screw M5x12 TORX 25
- 3 Pole clamp spring
- 4 Pole clamp lever

Disassembly

- 1. Loosen three screws (Abb.: 4 4 / Item 2) and remove screws together with the pole clamp guide (Abb.: 4 4 / Item 1).
- 2. Remove pole clamp lever (Abb.: 4 4 / Item 4) and pole clamp spring (Abb.: 4 4 / Item 3).

4 - 6 SpaceStation, 2.0 gb

4.6 Housing Back Panel

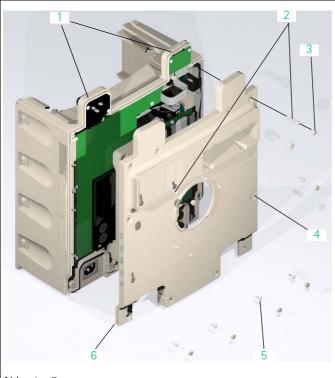


Abb.: 4 - 5

Legende zu Abb. 4 - 5: ItemDesignation

- 1 Housing notch
- 2 Screw EJOT 30x16 WN 5452 TORX 10IP A2
- 3 Cover cap
- 4 Blind plug
- 5 Screw EJOT 30x9 WN 5452 TORX 10IP A2
- 6 Housing back panel SPCO

Designation

Ord. No.

Cover cap and screws

(see "Service Parts and Screw Kit" ⇒ S. 4 - 2)

Disassembly

Note

The third cover cap on the right side of the housing back panel when viewed from above is a blind plug (Abb.: 4 - 5 / Item 4) which does not need to be removed.

- 1. Pierce ten cover caps (Abb.: 4 5 / Item 3) with a screwdriver and pull caps out.
- 2. Unscrew eight screws (Abb.: 4 5 / Item 5).
- 3. Unscrew two screws (Abb.: 4 5 / Item 2).

CAUTION


When you remove the housing back panel, pay attention to the connecting cables to the unit. They might get off.

4. Remove the back panel (Abb.: 4 - 5 / Item 6) carefully from the housing and fold aside; pay attention to the cable connections.

Note

Lift the housing back panel carefully at the bottom over the seat of the pole clamp guide, push the back panel over the housing notch (Abb.: 4 - 5 / Item 1), and fold aside.

SpaceStation, 2.0 gb 4 - 7

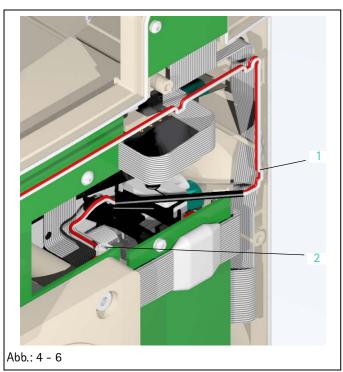

Legende zu Abb. 4 - 6:

ItemDesignation

- 1 Fan connecting cable (presented without back panel)
- 2 Connector

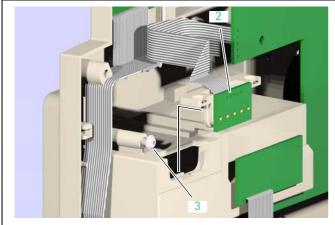


Abb.: 4 - 7

Legende zu Abb. 4 - 7:

ItemDesignation

- 1 Contact strip
- 2 Screw A2 WN 5452 30x9


5. Pull off connector (Abb.: 4 - 6 / Item 2) of the fan connecting cable (Abb.: 4 - 6 / Item 1) at the SPCO board.

6. Unscrew one screw (Abb.: 4 - 7 / Item 2) and remove the contact strip (Abb.: 4 - 7 / Item 1) carefully out of the housing back panel.

Note

If necessary, the spring-mounted contact pins must be carefully inserted when the contact strip is dismounted.

4 - 8 SpaceStation, 2.0 gb

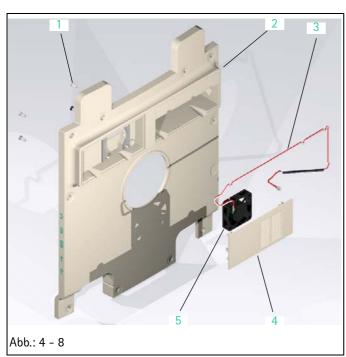

Legende zu Abb. 4 - 8:

ItemDesignation

- 1 Screw EJOT 30x9 WN 5452 TORX 10IP A2
- 2 Housing back panel SPCO
- 3 Fan connecting cable
- 4 Fan grid SPCO
- 5 Fan SPCO

Disassembly

 Designation
 Ord. No.

 Housing back panel SPCO
 3452 1052

 Fan SPCO
 3452 1048


Cover caps and screws

(see "Service Parts and Screw Kit" → S. 4 - 2)

- 1. Note and outline the cable layout of the fan connecting cable (Abb.: 4 8 / Item 3).
- 2. Unscrew four screws (Abb.: 4 8 / Item 1).
- 3. Remove fan grid (Abb.: 4 8 / Item 4) out of the housing back panel (Abb.: 4 8 / Item 2).
- 4. Remove fan (Abb.: 4 8 / Item 5) out of the housing back panel.

SpaceStation, 2.0 gb 4 - 9

4.7 Device Bracket

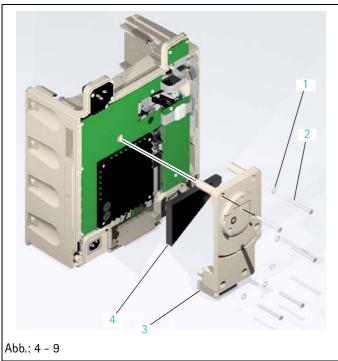

Legende zu Abb. 4 - 9:

ItemDesignation

- 1 Serrated lock washer M5
- 2 Screw M5x45 A2 DIN912 TORX 25
- 3 Device bracket
- 4 Insulating hood


Designation	Ord. No.
Device bracket SPS	3452 1127
Insulating hood SPCO	3452 1043
Screws	
(see "Service Parts and Screw Kit" ⇒ S. 4 - 2)	

Disassembly

- 1. Loosen five screws (Abb.: 4 9 / Item 2) and remove screws together with the serrated lock washers (Abb.: 4 9 / Item 1) and the device bracket (Abb.: 4 9 / Item 3).
- 2. Remove insulating hood (Abb.: 4 9 / Item 4) from the power supply.

4 - 10 SpaceStation, 2.0 gb

4.8 SPCO PCB

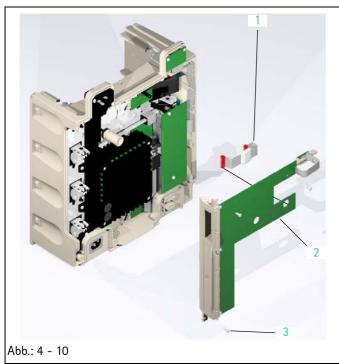

Legende zu Abb. 4 - 10:

Item Designation

- 1 SPCO connecting cable
- 2 SPCO PCB
- 3 Screw EJOT 30x9 WN 5452 TORX 10IP A2

Designation	Ord. No.
SPCO PCB	3452 1053
SPCO connecting cable	3452 1042
SPCO insulating piece	3452 1044
SPCO fastening sleeve	3452 1047
Screws	

(see "Service Parts and Screw Kit" → S. 4 - 2)


Disassembly

- 1. Unscrew two screws (Abb.: 4 10 / Item 3).
- 2. Remove SPCO PCB (Abb.: 4 10 / Item 2) carefully out of the housing and fold aside; pay attention to the length of the SPCO connecting cable (Abb.: 4 10 / Item 1).
- 3. Pull off the SPCO connecting cable on the SPCO PCB and remove the board out of the housing.
- 4. Loosen two screws of the interface board and lift the interface board carefully until the SPCO connecting cable can be pulled off.
- 5. Pull off the SPCO connecting cable at the interface board.

SpaceStation, 2.0 gb 4 - 11

Disassembly / Assembly SpaceCom



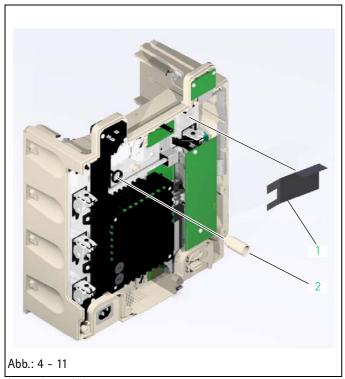

- Legende zu Abb. 4 11: ItemDesignation
- 1 SPCO insulating piece
- 2 SPCO fastening sleeve

- 6. Remove the fastening sleeve (Abb.: 4 11 / Item 2) out of the housing.
- 7. Remove the insulating piece (Abb.: 4 11 / Item 1) under the connector board F2A.

4.9 Further Disassembly

Further disassembly of the SpaceStation is continued with the power supply and corresponds to the disassembly of a SpaceStation with SpaceCom. This disassembly is described in Chapter "Power Supply" (S. 3 - 7) in more detail.

4 - 12 SpaceStation, 2.0 gb

4.10 Assembly / Installation

Assembly or installation of the modules and subsystems is done in reverse order of disassembly. Special steps to be observed are described hereafter in detail.

Only new cover caps are to be used.

Special Screws

Special screws for plastic housings are used in this unit. The screws are not self-cutting but produce a thread in the plastic of the housing through deformation when fitted in for the first time.

If the beginning of the thread is not engaged when the screw is fitted, a new thread is produced and the old thread is destroyed so that the security of the fixing can no longer be guaranteed.

Proceed as follows to fit the special screws:

- 1. Put the screw on the thread.
- 2. Rotate screw anti-clockwise (loosen) until a faint click can be heard. This click is produced when the screw thread drops into the existing thread.
- 3. Screw in the screw and tighten with the defined torque.

Housing Back Panel

CAUTION

When mounting the contact strip see that the springy contacts are not bent or broken.

- 1. Push the contact strip (Abb.: 4 12 / Item 2) in the groove (Abb.: 4 - 12 / Item 1) at the battery compartment of the housing back panel and tilt carefully into mounting position.
- 2. Secure the contact strip with a screw (Abb.: 4 12 / Item 3) against falling out.

Note

Screw the screw in up to a distance of about 2 mm from the screwed mandrel. This is the optimum spacing to clamp the contact strip.


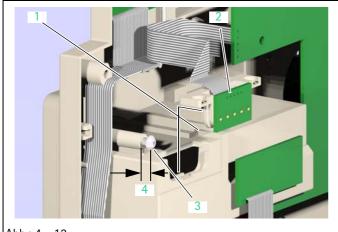


Abb.: 4 - 12

Legende zu Abb. 4 - 12: **ItemDesignation**

- Groove 1
- 2 Contact strip
- Screw A2 WN 5452 30x9
- Spacing: 2 mm

SpaceStation, 2.0 gb 4 - 13

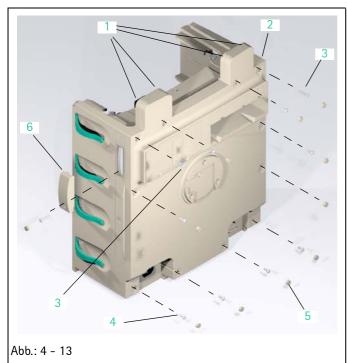

Legende zu Abb. 4 - 13:

ItemDesignation

2

- Locking tabs 1
- Housing back panel 3 Screw A2 WN 5452 30x16
- Screw A2 WN 5452 30x9 4
- 5 Housing cover cap
- W-LAN cover

Note

Before the housing back panel is definitely fitted check the insulating hood and the insulating piece for proper fit and the correct cable layout of the fan cable and of the connecting cable to the battery contact strip.

3. Place on the housing back panel (Abb.: 4 - 13 / Item 2) and fasten it with the two longer screws (Abb.: 4 - 13 / Item 3) and the eight shorter screws (Abb.: 4 - 13 / Item 4).

Note

Insert the housing back panel on the notches (Abb.: 4 - 13 / Item 1) at the two connector coverings of the housing and tilt the back panel in its final position.

Pay attention to the cable connections when placing on the housing back panel. These must not be squeezed.

Note

Pay attention to the different screw lengths when mounting the housing back panel. The longer screws are provided for the lefthand and right-hand upper screw opening.

4. No screw is inserted in the third mounting hole (from the top) on the right-hand side of the back panel. The hole must be covered with a cover cap.

4.11 Checks after Repair

- 1. Check the device to ensure safe functionality of the unit (see "Device Check" **⇒** S. 2 - 4).
- 2. Depending on the work carried out the specific steps of the TSC must be performed (see "Technical Safety Check (TSC)" →
- 3. The activities carried out must be recorded in the following check list.

4 - 14 SpaceStation, 2.0 gb

Check List for Checks after Repair

Visual Inspection		Electrical Safety		Functional Inspection
	acc	ording to IEC / EN 60601-1		
	or۱	/DE 0750 and VDE 0751		
Cleanliness		Mains voltage according to the TSC		Locking with second SPS
Completeness		Protective conductor resistance ac-		Pump locking of all slots
Damage and faults affecting safety		cording to the TSC		
Damage to and readability of the label		Earth leakage current according to the	Cor	nnect unit to mains.
Cover caps		TSC	Ger	neral functional check with SpaceCover
Connectors		Housing leakage current SPCO accord-	cor	nfort and pump of every slot.
Well running and sealing of the module		ing to the TSC		Fan SPCO (approx. 1 sec.)
lock				Status indicator (green) lights up
Mains cable				Alarm indicator (red) is on for
Accessories				approx. 1 min.
				Voltage supply of all slots
				Alarm transmission
				Staff call transmission
				Ethernet interface (RJ 45)
				RS232 interface
				Charge state of the battery module (display on the SpaceCover comfort) Switch-over mains / battery

SpaceStation, 2.0 gb 4 - 15

4

Disassembly / Assembly SpaceCom

For your information:	

4 - 16 SpaceStation, 2.0 gb

5.1 General

Remarks on Disassembly / Assembly

Before disassembling the unit, the system must be checked (see "Device Check" → S. 2 - 4) to isolate the part to be exchanged.

All necessary steps to disassemble or dismount the complete unit with all its subsystems and spare parts of the SpaceCover standard and the SpaceCover comfort are detailed in the following description. Steps that are not necessary can be skipped.

Note

All figures in this chapter depict the SpaceCover comfort. In general, these figures also apply to the SpaceCover standard.

Note

Special screws for plastic housings are used in this unit. Pay attention to the corresponding notes when you fit the screws.

Service Parts and Screw Kit

All small parts, such as cover caps, are contained in a SpaceStation service part kit.

Designation Ord. No.

with:

Release button

Release seal

Seal M-2K

Cover cap for housing

Cover cap for handle

Housing foot SPS

Mains seal F1A

Pole clamp lever

Pole clamp spring

O-ring 6.07 x 1.78

0-ring 12.0 x 2.0

SpaceStation, 2.0 gb 5 - 1

Disassembly / Assembly SpaceCover

All screws used in the device are included in a SpaceStation screw kit.

Designation Ord. No.

Screw EJOT 30x9 WN 5452 TORX 10IP A2

Screw EJOT 30x12 WN 5452 TORX 10IP A2

Screw EJOT 30x16 WN 5451 TORX 10IP A2

Screw M5x45 A2 DIN912 TORX 25

Countersunk screw M5x12 TORX 25

Fillister-head screw M5x12 EN-ISO14583 A2

Nut M4

Insulating washer M3

Washer M4


Washer M5

Serrated lock washer M4

Serrated lock washer M5


5 - 2 SpaceStation, 2.0 gb

5.2 Battery Compartment Cover

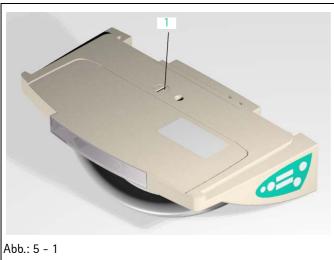

Legende zu Abb. 5 - 1: ItemDesignation

1 Battery compartment cover lock

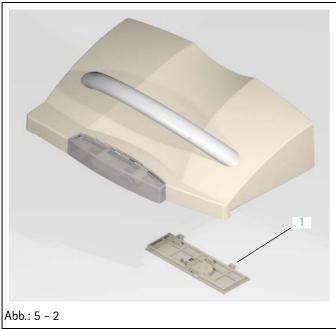

Legende zu Abb. 5 - 2: ItemDesignation

1 Battery compartment cover

Designation Ord. No.

Battery compartment cover SPC...... 3452 1232

Disassembly

Rotate the lock (Abb.: 5 - 1 / Item 1) on the battery compartment cover (Abb.: 5 - 2 / Item 1) and remove the battery compartment cover.


CAUTION

The battery module which might be installed in the SpaceCover comfort can drop out when the battery compartment cover is removed.

SpaceStation, 1.0 gb 5 - 3

5.3 Battery Module

(Only SpaceCover comfort)

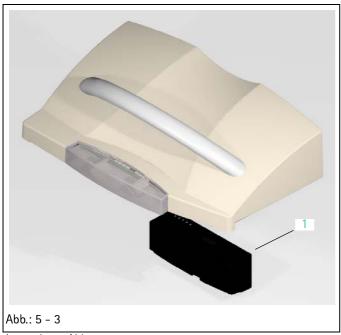

Legende zu Abb. 5 - 3: ItemDesignation

1 Battery module (option)


DesignationOrd. No.Battery pack SP (NIMH)0871 3180

Disassembly

1. Remove the battery module (option) (Abb.: 5 - 3 / Item 1) out of the battery compartment.

5 - 4 SpaceStation, 1.0 gb

5.4 Handle

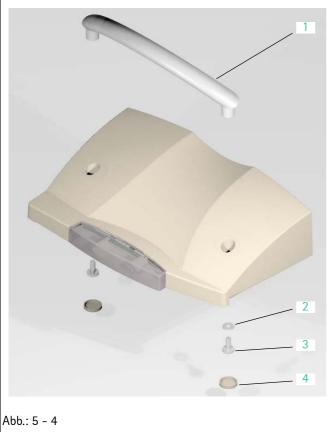

Legende zu Abb. 5 - 4:

Item Designation

- 1 Handle
- 2 Washer M5
- 3 Fillister-head screw M5x12 TORX
- 4 Cover cap for handle

Disassembly


1. Pierce two cover caps (Abb.: 5 - 4 / Item 4) on the housing bottom part with a screwdriver and pull caps out.

(see "Service Parts and Screw Kit" ⇒ S. 5 – 1)

- 2. Unscrew two fillister-head screws (Abb.: 5 4 / Item 3) and remove screws together with washers (Abb.: 5 4 / Item 2).
- 3. Pull the handle (Abb.: 5 4 / Item 1) upward and out the housing upper part.

SpaceStation, 2.0 gb 5 - 5

5.5 Housing Upper Part

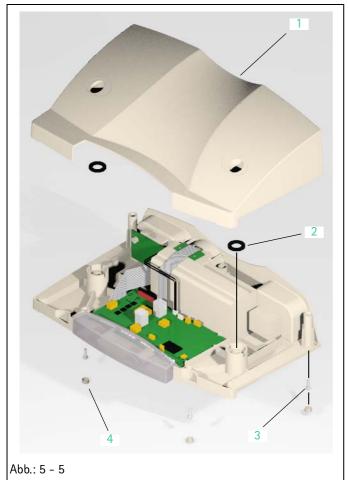

Legende zu Abb. 5 - 5: ItemDesignation

1 Housing upper part

- 2 Seal
- 3 Screw EJOT 30x9 WN 5452 TORX 10IP A2
- 4 Cover cap for housing

Designation	Ord. No.
Housing upper part SPCS	3452 1194
Housing upper part SPCC	3452 1178
Cover caps, screws and release seal	
(see Service Parts and Screw Kit" → S. 5 - 1)	

Disassembly

- 1. Pierce five cover caps (Abb.: 5 5 / Item 4) with a screwdriver and pull caps out.
- 2. Unscrew five screws (Abb.: 5 5 / Item 3) and remove the housing upper part (Abb.: 5 5 / Item 1) carefully.


Note

Pay attention to the length of the rubber cable between the operator panel and the processor PCB SCc when dismounting the housing upper part of the SpaceCover comfort.

Note

Do not remove the housing back panel together with the housing upper part.

5 - 6 SpaceStation, 1.0 gb

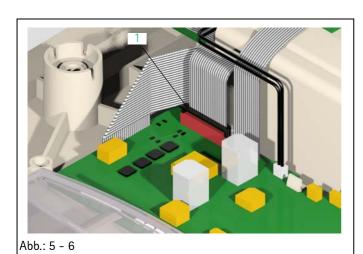


Legende zu Abb. 5 - 6: ItemDesignation

1 Connector lock

- 3. Pull the connector lock (Abb.: 5 6 / Item 1) on the processor PCB SCc upward.
- 4. Remove the rubber cable to the operator panel.
- 5. Remove two seals (Abb.: 5 5 / Item 2) in the housing bottom part.

SpaceStation, 1.0 gb 5 - 7

5.6 Housing Back Panel

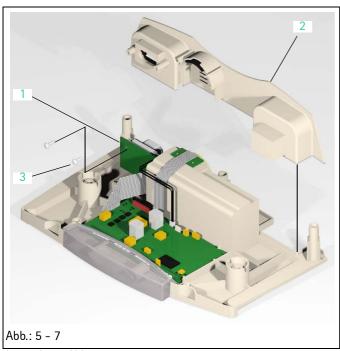

Legende zu Abb. 5 - 7: ItemDesignation

- 1 Connector board
- 2 Housing back panel
- 3 Screw EJOT 30x9 WN 5452 TORX 10IP A2

Designation	Ord. No.
Housing back panel SPC	3452 1216
Connector board SPCS	3452 1240
(only for SpaceCover standard)	

Screws

(see "Service Parts and Screw Kit" → S. 5 - 1)

Disassembly

1. Unscrew two screws (Abb.: 5 - 7 / Item 3) on the housing back panel and remove screws together with the connector board (Abb.: 5 - 7 / Item 1) (Abb.: 5 - 7 / Item 2).

Note


The connector board in the SpaceCover comfort is integrated in the SpaceCover comfort processor PCB.

2. Take the housing back panel out of the housing bottom part.

5 - 8 SpaceStation, 1.0 gb

5.7 Loudspeaker

(Only SpaceCover comfort)

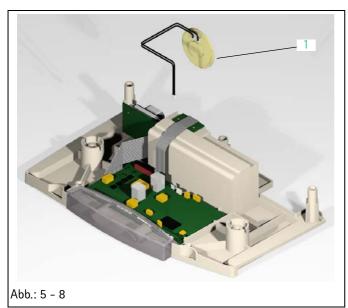

Legende zu Abb. 5 - 8: ItemDesignation

1 Loudspeaker

DesignationOrd. No.Loudspeaker SPCC3452 1267


Disassembly

- 1. Pull the loudspeaker connector (Abb.: 5 8 / Item 1) off the processor PCB.
- 2. Take the loudspeaker out of the housing bottom part.

SpaceStation, 1.0 gb 5 - 9

5.8 Processor PCB

(Only SpaceCover comfort)

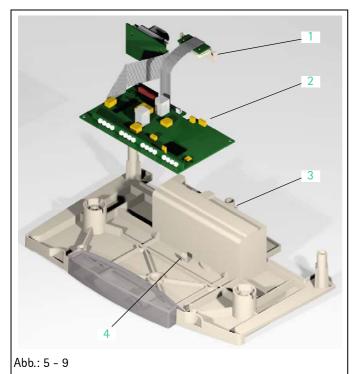

Legende zu Abb. 5 - 9:

ItemDesignation

- 1 Contact strip
- 2 Processor PCB
- 3 Housing bottom part
- 4 Release button

Designation	Ord. No.
Processor PCB SPCC	3452 1259
Housing bottom part SPCS	3452 1208
Housing bottom part SPCC	3452 1186
Disassembly	

1. Push the contact strip (Abb.: 5 - 9 / Item 1) to the battery module to the rear and out of the housing bottom part (Abb.: 5 - 9 / Item 3).

Note

The contact strip is integrated in the processor PCB.

2. Press the release button (Abb.: 5 - 9 / Item 4) in the housing bottom part carefully to the rear and remove the processor PCB (Abb.: 5 - 9 / Item 2) out of the housing bottom part.

5 - 10 SpaceStation, 1.0 gb

5.9 Assembly / Installation

Assembly or installation of the modules and subsystems is done in reverse order of disassembly. Special steps to be observed are described hereafter in detail.

Only new cover caps are to be used.

Special Screws


Special screws for plastic housings are used in this unit. The screws are not self-cutting but produce a thread in the plastic of the housing through deformation when fitted in for the first time.

If the beginning of the thread is not engaged when the screw is fitted, a new thread is produced and the old thread is destroyed so that the security of the fixing can no longer be guaranteed.

Proceed as follows to fit the special screws:

- 1. Put the screw on the thread.
- Rotate screw anti-clockwise (loosen) until a faint click can be heard. This click is produced when the screw thread drops into the existing thread.
- 3. Screw in the screw and tighten with the defined torque.

SpaceStation, 1.0 gb 5 - 11

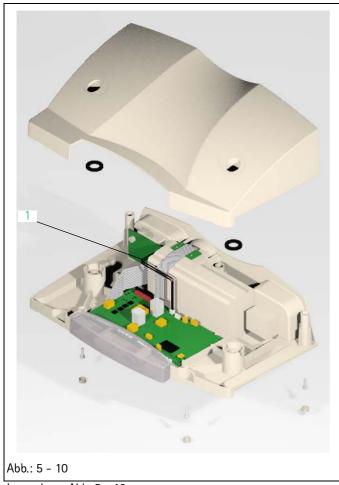

Housing Upper Part

Note

Pay attention to the correct position of the loudspeaker connecting leads when mounting the housing upper part, please see Abb.: 5 – 10. The leads must point upwards.

Legende zu Abb. 5 - 10: ItemDesignation

1 Loudspeaker connecting leads

5.10 Checks after Repair

- Check the device to ensure safe functionality of the unit (see "Device Check" → S. 2 - 4).
- Depending on the work carried out the specific steps of the TSC must be performed (see "Technical Safety Check (TSC)"

 S. 7 1).
- 3. The activities carried out must be recorded in the following check list.

5 - 12 SpaceStation, 1.0 gb

Check List for Checks after Repair

Visual Inspection	Electrical Safety		Functional Inspection
	according to IEC / EN 60601-1		
	or VDE 0750 and VDE 0751		
Cleanliness			Locking with SPS
Completeness			Battery compartment cover
Damage and faults affecting safety			
Damage to and readability of the label		Ger	neral functional check with the SpaceS-
Cover caps		tat	ion:
Connectors			Self-test
			Alarm
			Loudness setting
			Battery charge state
			Switch-over mains / battery
			Dimming function of the status- and
			alarm display line

SpaceStation, 2.0 gb 5 - 13

5

Disassembly / Assembly SpaceCover

F ' C ''	
For your information:	

5 - 14 SpaceStation, 1.0 gb

Cleaning

Clean and disinfect the SpaceStation at regular intervals. To clean the system we recommend mild soap-suds.

WARNING

WHILE CLEANING AND DISINFECTING THE SPACESTATION, DIS-CONNECT THE UNIT FROM THE MAINS SUPPLY.

CAUTION

Take care that no water or liquid enters the device between the housing front and back panel.

For disinfection by wiping, you should use for example Meliseptol from B. Braun. Allow the unit to dry for at least one minute. When you disinfect the device by spraying, make sure not to spray in the system openings (such as loudspeaker opening, interface connector).

The fan in the housing back panel of the SpaceStation with SpaceCom is to be vacuumed regularly. If the fan is badly soiled dismount the housing back panel and clean the fan and the area around the fan using a brush.

Servicing the Battery

The instructions for use contain a detailed description on how to service the battery.

SpaceStation, 2.0 gb 6 - 1

6 Servicing the Unit

F ' C ''	
For your information:	

6 - 2 SpaceStation, 1.0 gb

Technical Safety Check (TSC)

Index b (Master – to be added to the documentation)

Checklist for Technical Safety Check – Every 24 Months				
Unit: SpaceStation (SPS) / SpaceStation with SpaceCom (SPCO) / SpaceCover standard (SPCS) / SpaceCover comfort (SPCC) Manufacturer: B. Braun Melsungen AG Observe the Service Manual and the instructions for use. All measured values are to be documented. Accessories used should be included in testing. Make excl. use of calibrated measuring equipment. Article No. Unit No. Year of Procurement Year of Procurement				
Visual Inspection	Electrical Safety	Functional Inspection		
☐ Cleanliness, completeness, damage and legibility of the labels	as per EN 60601-1 or VDE 0750 Part 1 and VDE 0751 Part 1 Mains voltage V AC Protective conductor resistance Ω	Switch on unit. Staff call		
□ Pump suspension □ Well running and sealing of the pump	Unit incl. mains connecting line to mains voltage	☐ Indicator lamps SPCC ☐ Audible alarm on SPCC ☐ Indicator lamps SPCO		
lock Data and power interface (MSC connector) Interfaces SPCO Mains cable / connector and connecting line	output F1B ≤ 0.2Ω □ Earth leakage current at NC μA ≤ 30 μ A □ Earth leakage current at SFC μA ≤ 60 μ A □ Housing leakage current SPCO μA Metal collar Ethernet RJ45 ≤ 25 μA without SL	☐ Indicator lamps SPCO ☐ Functional inspection according to the instructions for use and dependent on the configuration		
	Accessories Used	Units Used		
	o	☐ Perfusor® Space, serial No.: ————————————————————————————————————		
Test result:		Inspection performed by:		
Defects found which could endanger patien Measures to be taken: Repair ———————————————————————————————————	Unit handed over on: To:			
Special features / documentation:		Date / Signature: Next deadline:		
		Next dedullife.		



Technical Safety Check (TSC)

Index b (Master – to be added to the documentation)

For your information:	



Visual Inspection

- 1. Check the unit and accessories for cleanliness.
- 2. Check the unit and accessories for completeness and check configuration.
- 3. Check the unit and its accessories for damage and the labels for readability. Pay special attention to the following parts:
 - a) Pump suspension
 - b) Tightness of pump locks
 - c) Connectors
 - d) 8 tube guides
 - e) Mains cable

Electrical Safety according to IEC/EN 60601-1 or VDE 0750 and VDE 0751 The values to be measured for the electrical safety are stated in the TSC (see "Technical Safety Check (TSC)" \Rightarrow S. 7 - 1).

Protective Conductor Resistance

Note

The protective conductor resistance of each SpaceStation must be measured separately. Before carrying out the measurement remove all interface cables to the other units.

Measure between the protective conductor of the mains cable and the following parts:

 Ground contact in connector F1B (middle pin)

SpaceStation, 2.0 gb 8 - 1

Earth Leakage Current

Note

Due to the technical design the patient- and housing leakage current is not to be measured on the SpaceStation without SpaceCom and without pump inserted. The earth leakage current is always measured as it is the upper limit of leakage currents which can be measured.

The earth leakage current of each SpaceStation must be measured separately.

Check

- 1. Before carrying out the measurement remove all interface cables to the other units.
- 2. Remove all pumps out of the SpaceStation.
- 3. Measure the earth leakage current without single fault condition (NC = Normal Condition) incl. mains cable.
- 4. Document value.
- 5. Measure the earth leakage current in single fault condition (SFC = Single Fault Condition) incl. mains cable.
- 6. Document value.

8 - 2 SpaceStation, 2.0 gb



Legende zu Abb. 8 - 1: ItemDesignation

1 Metal frame of the Ethernet (RJ 45) connection

Housing Leakage Current (with the SpaceStation with SpaceCom)

Measurement is to be carried out at nominal voltage +10% in single fault condition (SFC = Single Fault Condition) incl. mains cable.

1. Measure the housing leakage current between mains input at the mains cable and the metal frame of the Ethernet (RJ 45) connection in the SpaceCom plug strip.

Note

The metal frames of the connectors in the SpaceCom plug strip are connected with each other.

2. Document value.

SpaceStation, 2.0 gb 8 - 3

Functional Inspection

Mechanical

- 1. Check proper functioning of the lock of all pump slots with one pump.
- 2. Check the battery compartment cover lock of the SpaceStation with SpaceCom for proper operation.
- 3. Check proper functioning of the lock to another SpaceStation or a SpaceCover.
- 4. Check the battery compartment cover lock of the SpaceCover comfort for proper operation.

Functional Check

- 1. Connect the SpaceStation to the mains.
- 2. In case of a SpaceStation with SpaceCom the following functions are to be checked:
 - Fan in the housing back panel starts running for appr. 1 sec.
 - b) Status indicator (green) on the SpaceCom lights up
 - c) Alarm indicator (red) on the SpaceCom lights up for approx. 1 minute
- 3. Trigger an alarm with a pump at each slot. The red LED on the service connector SP must light up.
 - When a SpaceCover comfort is fitted the alarm must also be displayed in the status- and alarm display line of the Space-Cover comfort; an audible alarm must sound.
- Carry out a self-test of the SpaceCover comfort (actuate the On- / Off button on theSpaceCover comfort), please see instructions for use.

Battery Check of the SpaceCover comfort

- Insert a battery module in the SpaceCover comfort. After start-up the battery charge condition is displayed in the top charge condition display of the operator and status display field of the SpaceCover comfort.
- Disconnect the SpaceStation from the mains. An audible alarm sounds and the status display colour on the operator and status display field of the SpaceCover comfort changes from green to yellow.
- 3. Possible displays are shown on the SpaceCover comfort.
- 4. Switch the SpaceStation or the SpaceStation column on and then off via the SpaceCover comfort. A self-test is carried out.

8 - 4 SpaceStation, 2.0 gb

 Reconnect the SpaceStation to the mains. The status display colour on the operator and status display field of the Space-Cover comfort changes from yellow to green.

Interface Check of the SpaceStation with SpaceCom

- 1. Insert a pump in the SpaceStation.
- Connect a PC to the SpaceStation via the interface cable RJ-45 cross-over and start the Space-Online Program (see "SpaceOnline"

 S. 1 - 17).
- 3. Check in the Service Part whether the correct pump type and its On-/Off-status are displayed.

Note

If the IP-address of the SpaceStation is unknown, all parameters can be reset to the factory settings from software version 695E010001 on via a USB-stick with the "factorydefault" directory saved here. When the word "factorydefault" is read automatically resetting of the parameters in the SpaceStation is started. The IP-address of the SpaceStation is set to 192.168.100.41.

- Connect a PC with the SpaceStation via the interface cable RS232 SPCO (crossed) and start the BCCshow Program (see "BCCshow" ➤ S. 1 - 19).
- 5. Check whether the correct pump and its status are displayed on the PC.

Battery Check of the SpaceStation with SpaceCom

- Insert one battery module in the SpaceStation and one in the SpaceCover comfort. After start-up the battery charge condition is displayed in the bottom charge condition display of the operator and status display field of the SpaceCover comfort.
- 2. Remove the battery module out of the SpaceCover comfort.
- Disconnect the SpaceStation from mains when the status indicator (green) on the SpaceCom (SpaceCom is operative) lights up.
- 4. The status indicator (green) on the SpaceCom remains on for at least 3 seconds. The alarm indicator (red) must not light up or flash.
- 5. Remove the optional battery modules which are not part of the unit out of the device.

SpaceStation, 2.0 gb 8 - 5

8

Procedural Instructions on the TSC

For your information:	

8 - 6 SpaceStation, 1.0 gb

Test Equipment	Designation	Ord. No.
	For Device Check	
	Service-CD	0871 3301
	Service connector Space	3452 1062
	Interface cable RS232 SP	0871 3234
	Interface cable RS232 SPCO (crossed)	0871 3237
	Interface cable RJ45 cross-over	
	For Repairs	
	No special test equipment required.	
	For the TSC	
	- Measuring instrument for electrical safety	
	- Service connector Space (see above)	

Special ToolsDesignationOrd. No.TORX screwdriver kit (standard)
5 - 10, 25TORX screwdriver kit (Plus)
5 - 10, 25Screwdriver 6lPx60 TORX plus4002 4806Screwdriver 8lPx60 TORX plus4002 4814Screwdriver 10lPx60 TORX plus4002 4822Screwdriver 25lPx60 TORX plus4002 4830Torque wrench with TORX adapter

9

Test Equipment and Special Tools


For your information:	

9 - 2 SpaceStation, 1.0 gb

SpaceStation, SpaceCover


Service part kit SpaceStation 3477 4335 with: Release button Release seal Seal M-2K Cover cap for housing Cover cap for handle Housing foot SPS Seal, mains F1A Pole clamp lever Pole clamp spring 0-ring 6.07 x 1.78 O-ring 12.0 x 2.0 Screw kit SpaceStation 3477 4343 Screw EJOT 30x9 WN 5452 TORX 10IP A2 Screw EJOT 30x12 WN 5452 TORX 10IP A2 Screw EJOT 30x16 WN 5451 TORX 10IP A2 Screw M5x45 A2 DIN912 TORX 25 Countersunk screw M5x12 TORX 25 Fillister-head screw M5x12 EN-ISO14583 A2 Nut M4 Insulating washer M3 Washer M4 Washer M5 Serrated lock washer M4 Serrated lock washer M5

SpaceStation without SpaceCom

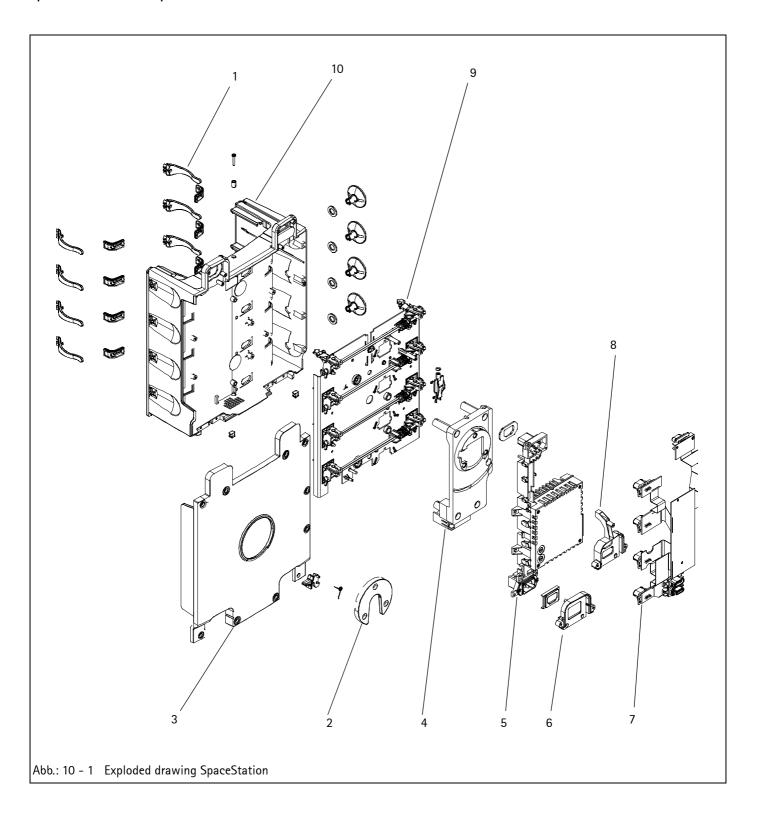


1	Tube guide SPS
2	Pole clamp guide SPS
3	Housing back panel SPS
4	Device bracket SPS
5	Power supply SPS
6	Connector holder, mains, SPS 3452 1143
7	Interface board SPS
8	Connector holder, data, SPS 3452 1160
9	Module lock SPS
10	Housing SPS

SpaceStation with SpaceCom

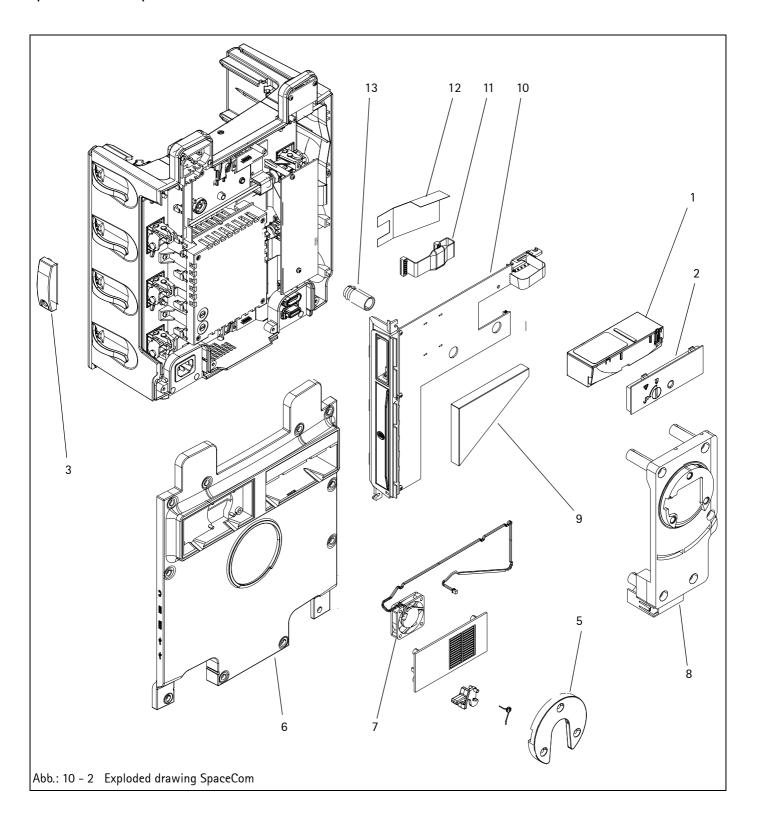

1	Battery pack SP (NIMH)
2	Battery compartment cover SPC
3	W-LAN cover SPC0
	W-LAN cover SPCO
4	Tube guide SPS
5	Pole clamp guide SPS
6	Housing back panel SPC0
7	Fan SPC0
8	Device bracket SPS
9	Insulating hood SPCO
10	SPCO PCB
11	SPCO connecting cable
12	SPCO insulating piece
13	SPCO fastening sleeve 3452 1047

SpaceCover

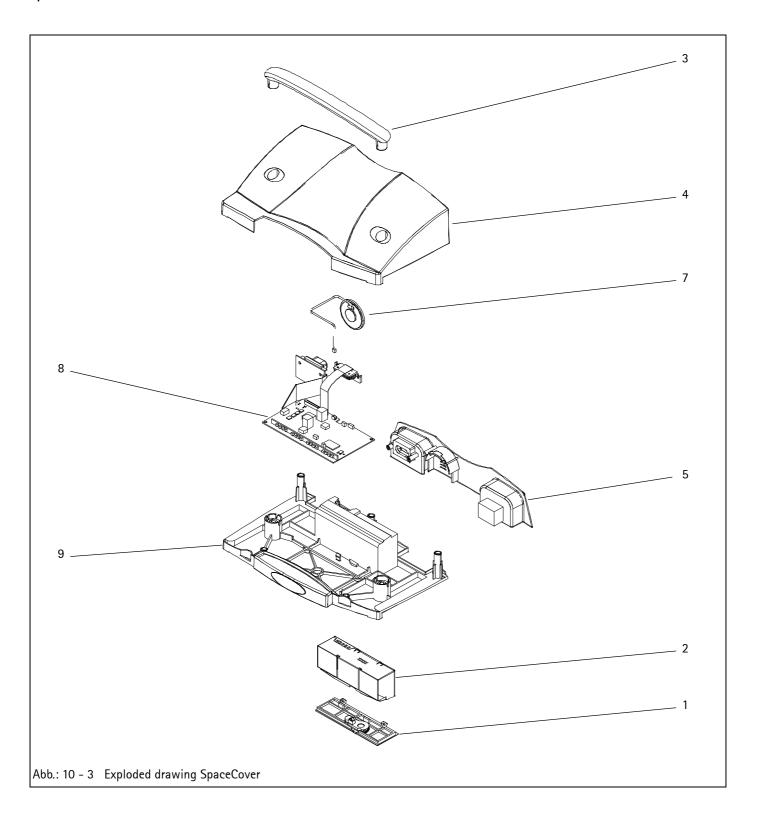

10 - 6SpaceStation, 1.0 gb

1	Battery compartment cover SPC
2	Battery pack SP (NIMH)
3	Handle SPC
4	Housing upper part SPCS
	Housing upper part SPCC
5	Housing back panel SPC
6	Connector board SPCS
7	Loudspeaker SPCC
8	Processor PCB SPCC
9	Housing bottom part SPCS
	Housing bottom part SPCC

Spare Parts List

10 - 8 SpaceStation, 1.0 gb

Description of Version

Version 1.0 (Base)

- First version of this Service Manual.
- Release date: September 20, 2005.

Version 2.0

- Insertion of the SpaceStation version with SpaceCom incl. new spare parts.
- Changed TSC for the SpaceStation.
- Texts corrected
- Release date: December 14, 2006.

Version List of the Individual Pages

11 Revision Documentation

11 - 2 SpaceStation, 1.0 gb

Title and Leading Pages	Page 1 - 22 Revision 1.0
Page 0 - 1	
Page 0 - 2 Revision 2.0	Chapter 2 "Unit Diagnosis / Calibration"
Page 0 - 3 Revision 2.0	Page 2 - 1 Revision 2.0
Page 0 - 4 Revision 2.0	Page 2 - 2 Revision 1.0
Page 0 - 5 Revision 2.0	Page 2 - 3 Revision 2.0
Page 0 - 6 Revision 2.0	Page 2 - 4 Revision 2.0
Page 0 - 7 Revision 2.0	Page 2 - 5 Revision 2.0
Page 0 - 8 Revision 2.0	Page 2 - 6 Revision 2.0
Page 0 - 9 Revision 2.0	Page 2 - 7 Revision 2.0
Page 0 - 10 Revision 2.0	Page 2 - 8 Revision 2.0
Page 0 - 11 Revision 2.0	Chapter 3 "Disassembly / Assembly SpaceStation"
Page 0 - 12 Revision 2.0	Page 3 - 1 Revision 2.0
	Page 3 - 2 Revision 2.0
Chapter 1 "System Overview"	Page 3 - 3 Revision 1.0
Page 1 - 1	Page 3 - 4 Revision 1.0
Page 1 - 2	Page 3 - 5 Revision 1.0
Page 1 - 3	Page 3 - 6
Page 1 - 4	Page 3 - 7
Page 1 - 5	Page 3 - 8
Page 1 - 6	Page 3 - 9
Page 1 - 7 Revision 2.0	Page 3 - 10
Page 1 - 8 Revision 2.0	Page 3 - 11
Page 1 - 9 Revision 2.0	Page 3 - 12
Page 1 - 10 Revision 1.0	Page 3 - 13
Page 1 - 11 Revision 1.0	Page 3 - 14
Page 1 - 12 Revision 2.0	Page 3 - 15
Page 1 - 13 Revision 1.0	Page 3 - 16 Revision 1.0
Page 1 - 14 Revision 2.0	-
Page 1 - 15 Revision 2.0	Page 3 - 17
Page 1 - 16 Revision 2.0	Page 3 - 18
Page 1 - 17 Revision 2.0	Page 3 - 19
Page 1 - 18 Revision 2.0	Page 3 - 20
Page 1 - 19 Revision 2.0	Page 3 - 21
Page 1 - 20 Revision 2.0	Page 3 - 22
Page 1 - 21 Revision 2.0	Page 3 - 23
	Page 3 - 24 Revision 1.0

Page 3 - 25 Revision 1.0	
Page 3 - 26 Revision 1.0	Chapter 6 "Servicing the Unit"
	Page 6 - 1 Revision 2.0
Chapter 4 "Disassembly / Assembly SpaceCom"	Page 6 - 2 Revision 1.0
Page 4 - 1 Revision 2.0	Chantan 7 "Taskmias Safatu Chask (TSC)"
Page 4 - 2 Revision 2.0	Chapter 7 "Technical Safety Check (TSC)"
Page 4 – 3 Revision 2.0	Page 7 - 1
Page 4 - 4 Revision 2.0	Page 7 - 2 Revision 1.0
Page 4 - 5 Revision 2.0	Chapter 8 "Procedural Instructions on the TSC"
Page 4 - 6 Revision 2.0	Page 8 – 1 Revision 2.0
Page 4 - 7 Revision 2.0	Page 8 - 2 Revision 2.0
Page 4 - 8 Revision 2.0	Page 8 - 3
Page 4 - 9 Revision 2.0	Page 8 - 4
Page 4 - 10 Revision 2.0	Page 8 - 5
Page 4 - 11	Page 8 - 6
Page 4 - 12 Revision 2.0	ruge o o Hevision no
Page 4 - 13 Revision 2.0	Chapter 9 "Test Equipment and Special Tools"
Page 4 - 14 Revision 2.0	Page 9 - 1 Revision 2.0
Page 4 - 15 Revision 2.0	Page 9 - 2 Revision 1.0
Page 4 - 16 Revision 2.0	Chapter 10 "Spare Parts List"
Chapter 5 "Disassembly / Assembly SpaceCover"	Page 10 - 1
Page 5 - 1 Revision 2.0	Page 10 - 2 Revision 2.0
Page 5 - 2 Revision 2.0	Page 10 - 3 Revision 1.0
Page 5 - 3 Revision 1.0	Page 10 - 4
Page 5 - 4 Revision 1.0	Page 10 - 5
Page 5 - 5 Revision 2.0	Page 10 - 6
Page 5 - 6 Revision 1.0	Page 10 - 7 Revision 1.0
Page 5 - 7 Revision 1.0	Page 10 - 8 Revision 1.0
Page 5 - 8 Revision 1.0	
Page 5 - 9 Revision 1.0	Chapter 11 "Revision Documentation"
Page 5 - 10 Revision 1.0	Page 11 - 1 Revision 2.0
Page 5 - 11	Page 11 - 2 Revision 1.0
Page 5 - 12 Revision 1.0	Page 11 - 3 Revision 2.0
Page 5 - 13 Revision 2.0	Page 11 - 4 Revision 2.0
Page 5 - 14 Revision 1.0	Page 11 - 5 Revision 2.0
	Page 11 - 6 Revision 1.0

Chapter 12 "Index"

Page 12 - 1	 Revision 2.0
Page 12 - 2	 Revision 2.0

11 - 6 SpaceStation, 1.0 gb

A	Housing 3 - 20
Accessories	Housing back panel 4 - 7
Alarms	Assembly
Assembly 3 - 1, 3 - 21, 4 - 1, 4 - 13, 5 - 1, 5 - 11	Disassembly
В	Housing back panel SPC
Battery compartment	Housing upper part
Battery compartment cover 4 - 3, 5 - 3	Assembly
Battery module	Disassembly 5 – 6
Blind plug	1
Block diagram 1 - 7, 1 - 9, 1 - 11	Installation
	Insulating hood
С	Insulating piece
Checks after repair	Interface board
SpaceCover 5 - 12	Disassembly
SpaceStation	Installation 3 - 21
Cleaning	
Contact strip	K
Current versions 0 - 5	Kontakte
D	L
Description	List of abbreviations
Device bracket	Loudspeaker 5 - 9
Device check	M
Disassembly	Module lock
E	Disassembly
Error codes	Installation
Error modes	
	0
F	Options
Fan	Р
Fan connecting cable	Pole clamp guide 3 - 4, 4 - 6
Fan grid	Power supply
Fastening sleeve	Processor PCB
Function	D.
Н	R
Handle	Release button

Index

Responsibilities	Special tools 9 - 1
Revision service	System overview
S	Т
Service Program	Technical data
Servicing the unit	Technical Safety Checks 0 - 5, 7 - 1
Software 1 - 12	Test equipment
Software update 1 - 13	Trouble shooting 2 - 6
SpaceCom 4 - 1	TSC 0 - 5, 7 - 1
SpaceStation	Procedural instructions 8 - 1
SpaceCom	Test equipment 9 - 1
Spare parts list	Tube guide
SPCO connecting cable	
SPCO PCB	W = I A N module 4 = 4