Technical Data

DU 5308 X-ray Tube

453570013501

453575073168 Rev A

DUNLEE

**X-ray Tube Technical Data
3.9 MJ/5.26 MHU CT Insert
0.5 x 0.7/0.8 x 1.2 Focal Spots**

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This X-ray Tube will be used as part of a registered and controlled X-Ray Tube Housing Assembly. Please refer to corresponding X-Ray Tube Housing Assembly document for important safety warnings, servicing and operating instructions, warranty and return procedures associated with installation, use and maintenance of this product.

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Copyright

© 2004 DUNLEE

All rights reserved. Reproduction in whole or in part is prohibited without the prior written consent of the copyright-holder. DUNLEE reserves the right to make changes in specifications or to discontinue any product, at any time without notice or obligation, and is not liable for any consequences resulting from the use of this publication.

Equipment is subject to change without notice. All changes will be in compliance with regulations governing manufacture of medical equipment. Printed in the USA. Document originally drafted in English.

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DISPOSAL

Dunlee Manufactures state-of-the-art x-ray equipment in terms of safety and environmental protection. Under normal operation, there are no risks to persons or the environment. To comply with regulations, it is necessary to use materials that may be harmful to the environment and therefore must be disposed of in a proper manner. Contact your Dunlee sales and service organization for proper handling and disposal.

This metal x-ray tube may contain beryllium. Disposal of this product must be through appropriate channels. (This statement is valid only for metal x-ray tubes, please disregard for glass tubes).

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RECYCLING STATEMENT

Dunlee strives to control manufacturing processes and environmental pollution. Consequently, Dunlee recycles select materials and components from X-ray Tubes. Elaborate testing and quality assurance procedures, including detailed inspection of all subcomponents, ensures the same high level of quality and functionality provided by new materials is maintained.

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INSTALLATION AND ASSEMBLY

The X-ray Tube must be installed into an appropriate x-ray tube housing assembly for safe and proper use. Only trained and qualified personnel must perform this final assembly and be performed in accordance with the documentation and guidelines provided by Dunlee. Failure to follow the appropriate procedure for final assembly will void the warranty.

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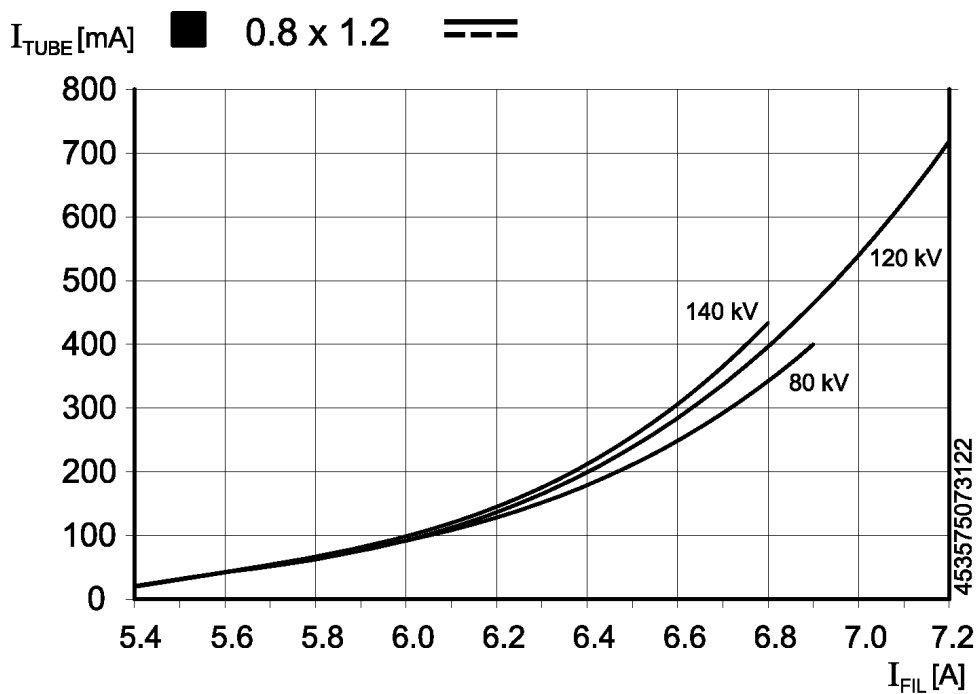
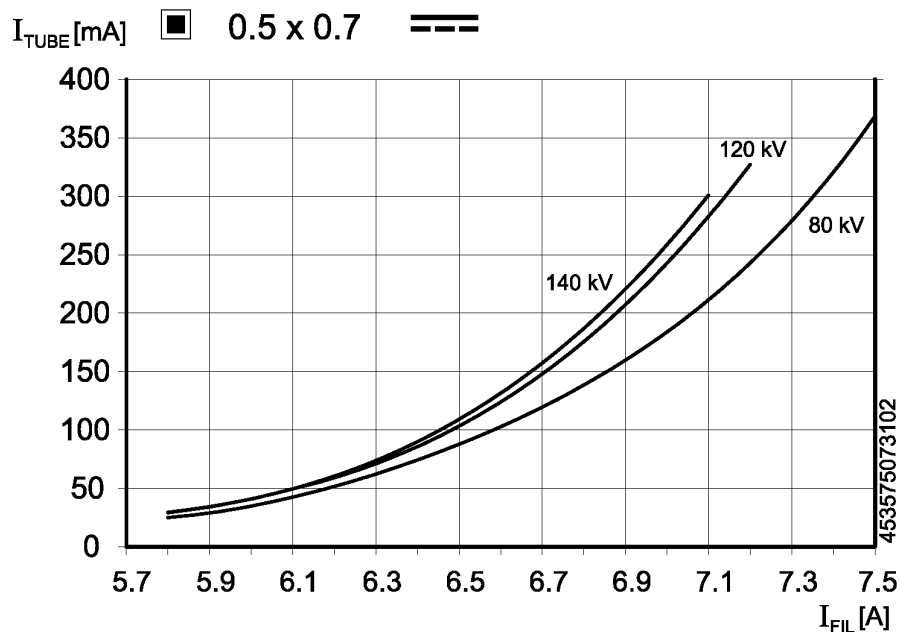
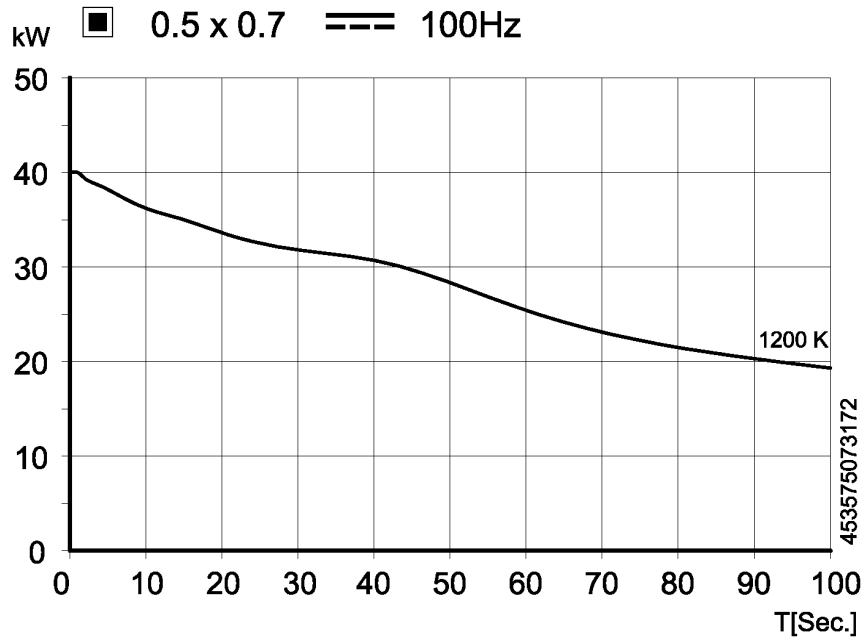
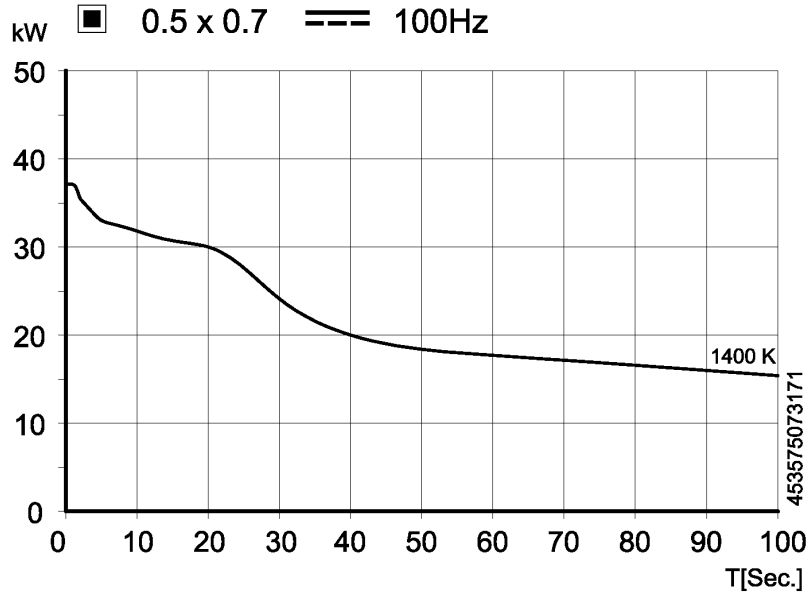
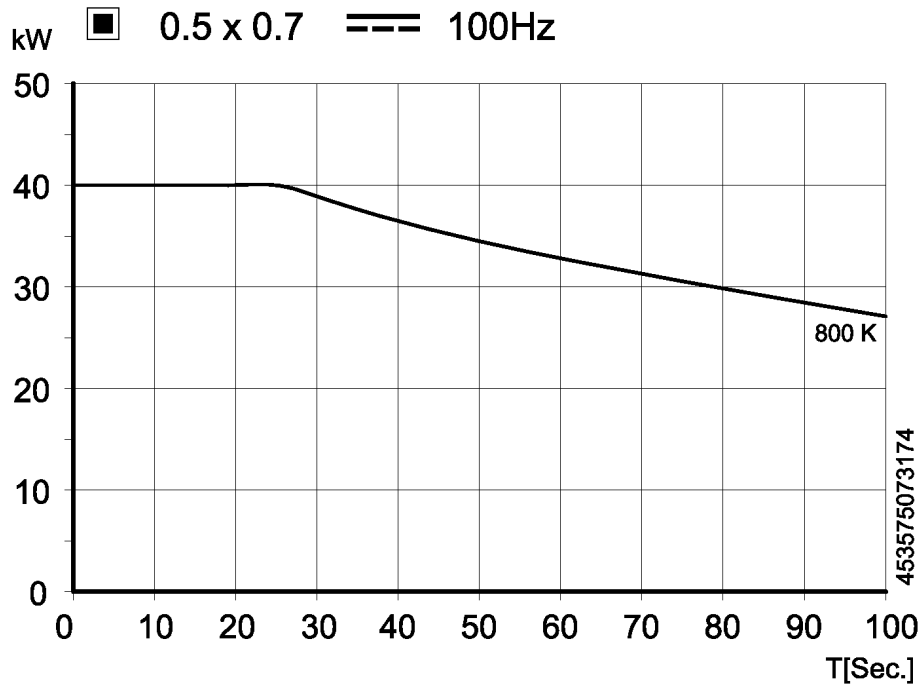
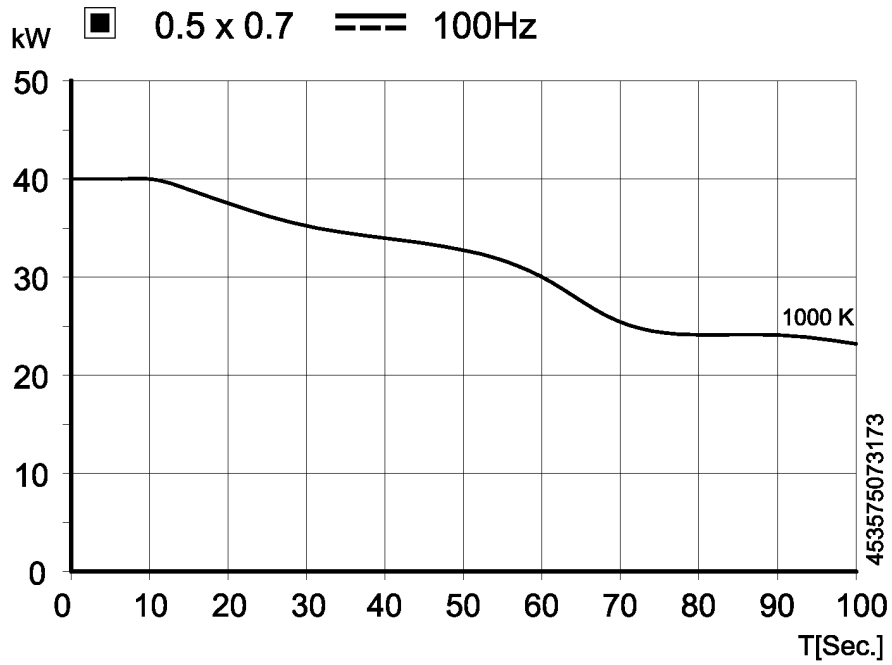
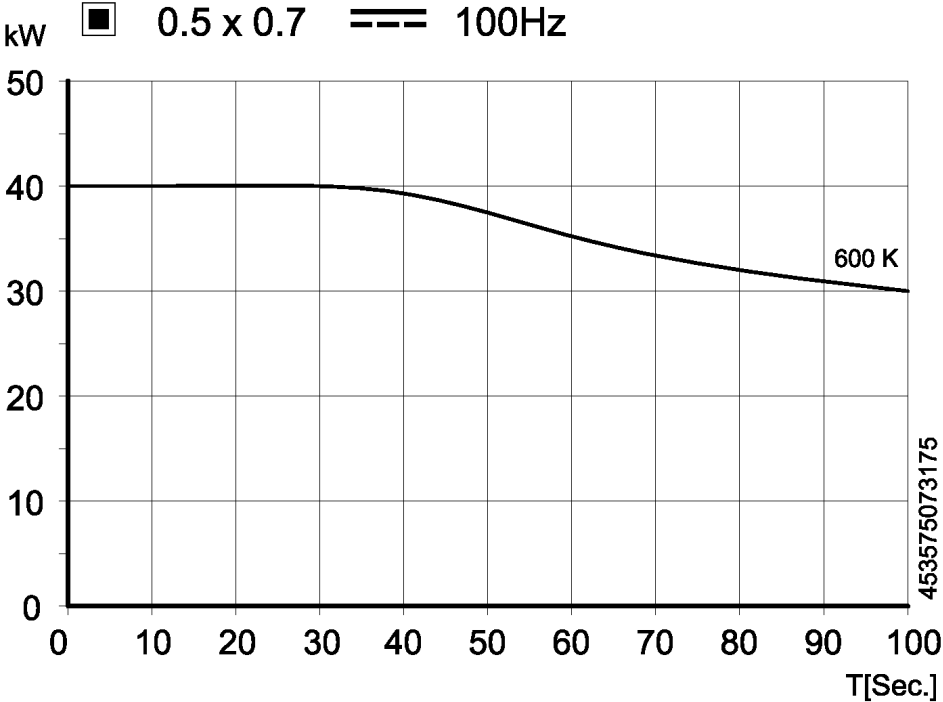
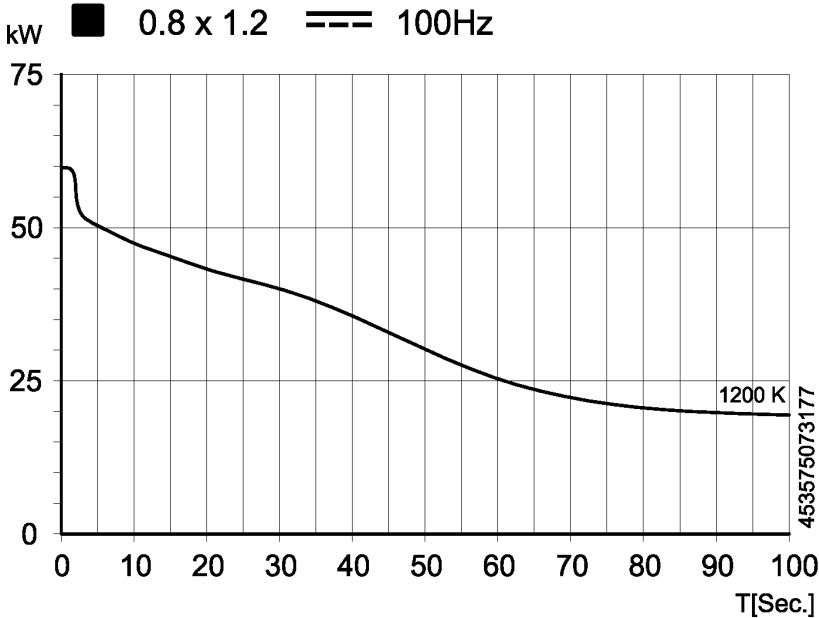
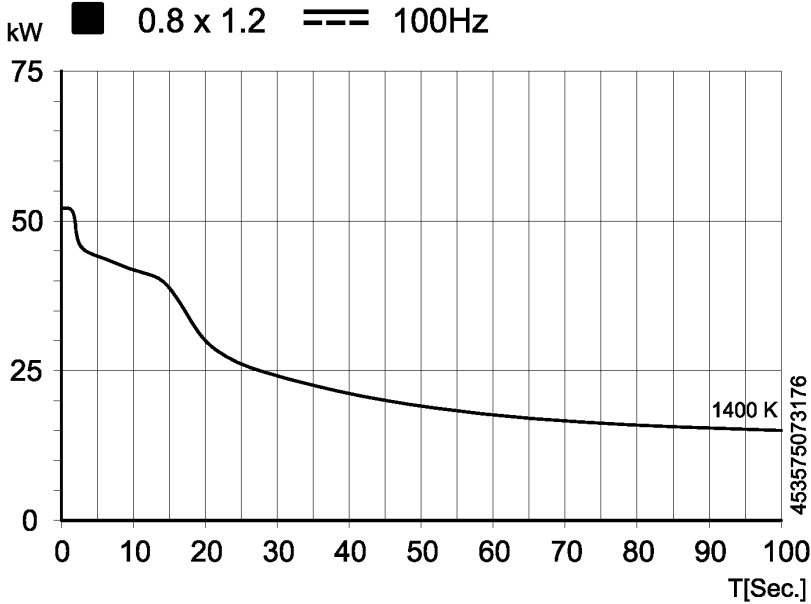
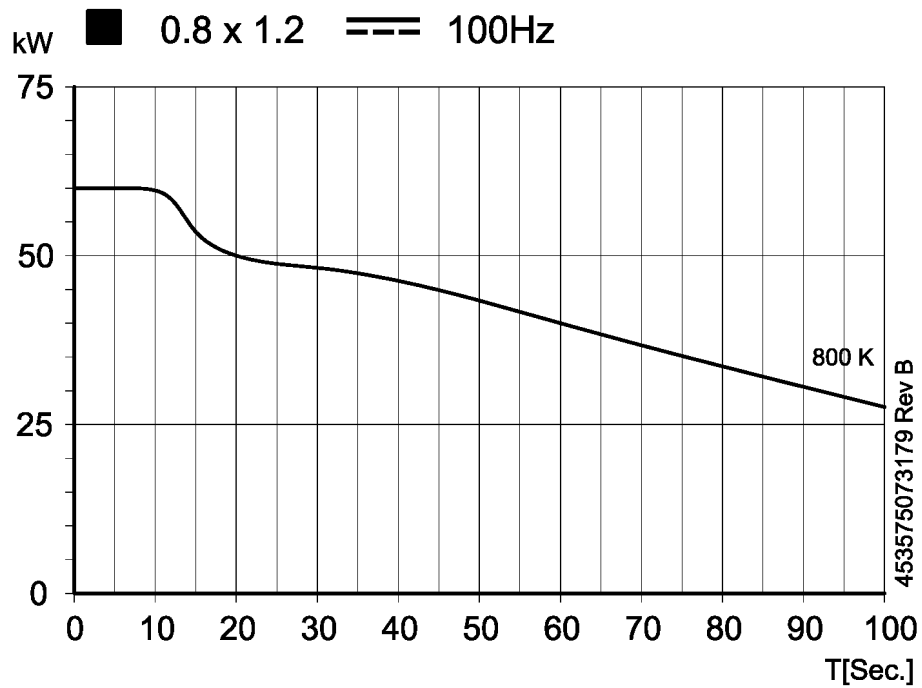
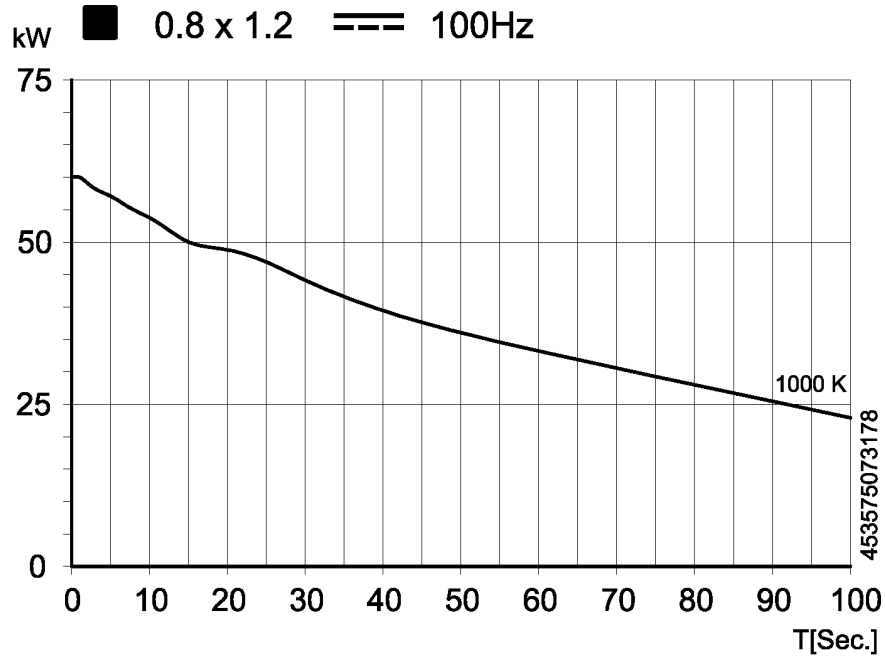
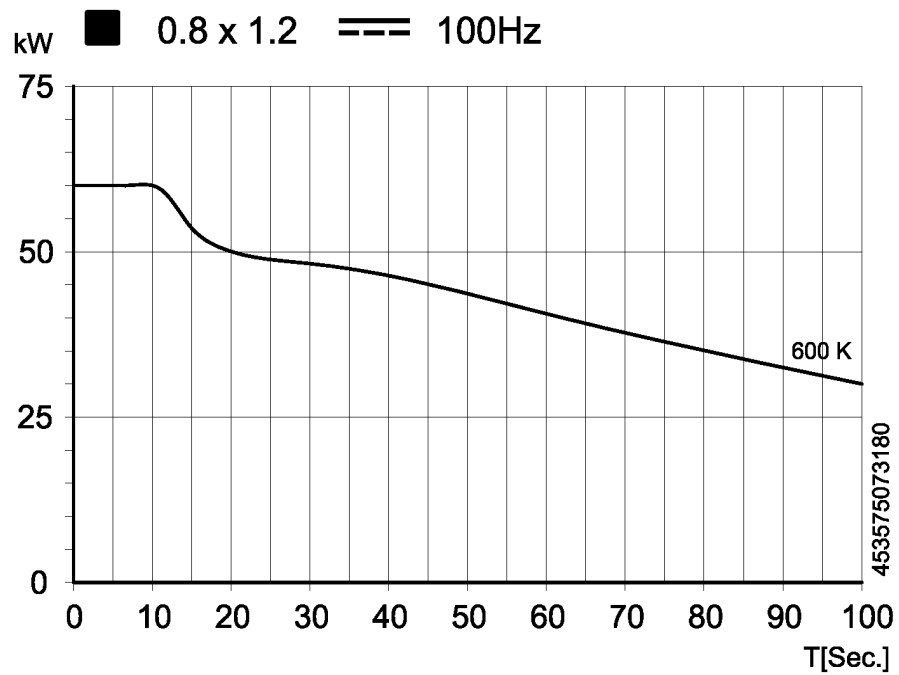
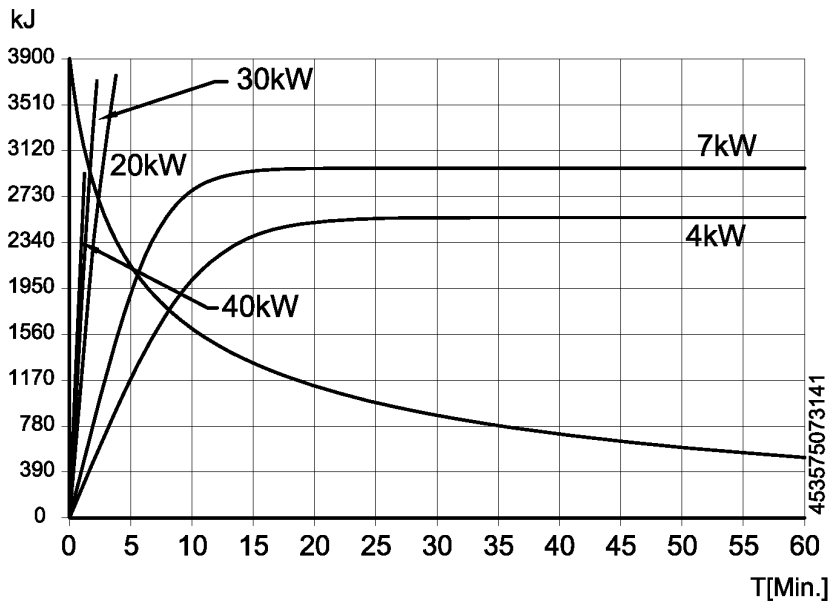
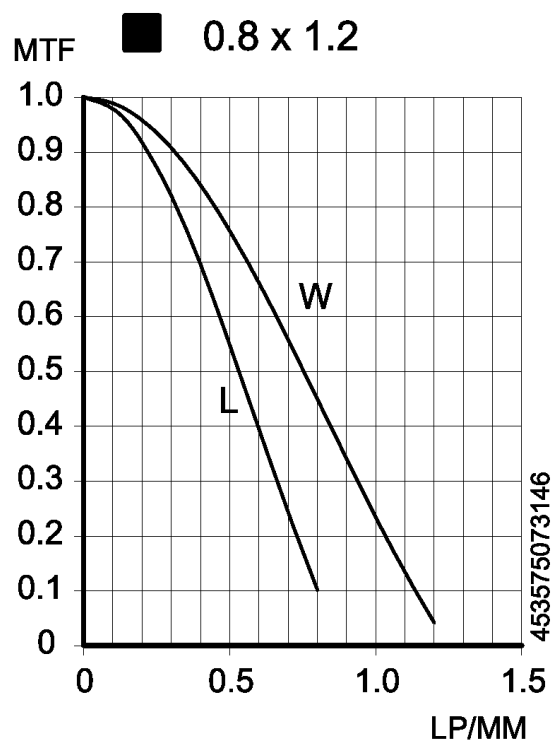
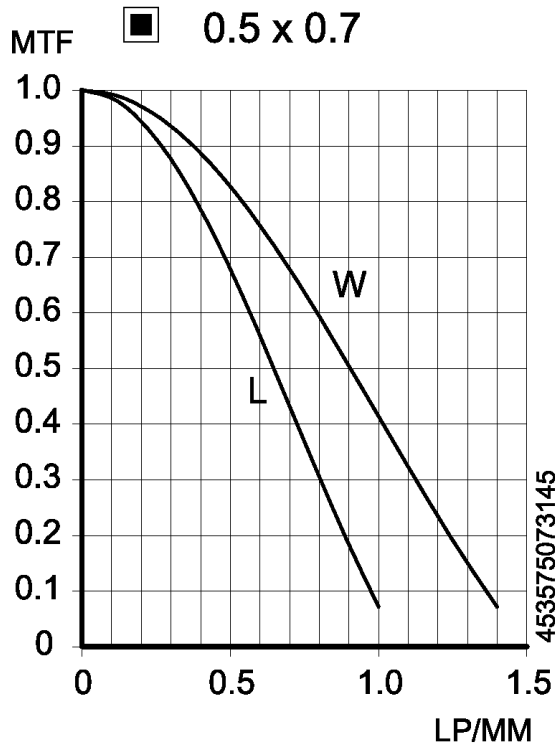
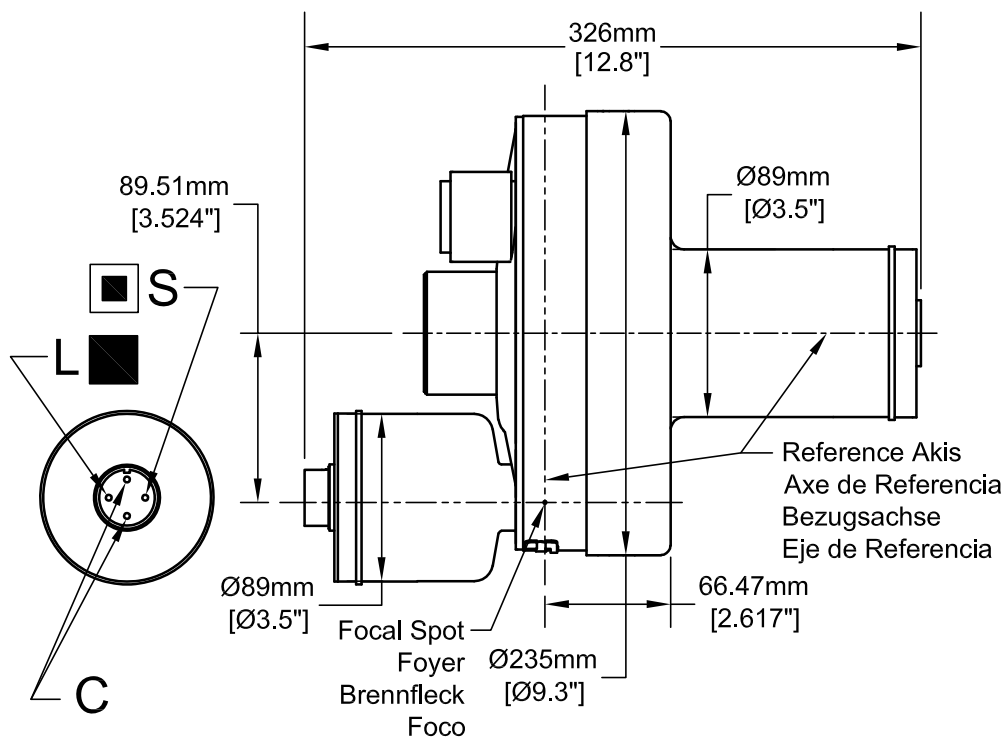
453575073167 Rev A

Section 8:

Service and Manufacturing Locations

Dunlee World Headquarters

555 North Commerce Street
Aurora, Illinois 60504 USA
Tel: 800.238.3780 (U.S. & Canada)
630.585.2100
Fax: 630.585.2125

Dunlee Tubemaster Facility

2312 Avenue J
Arlington, Texas 76006 USA
Tel: 800.544.9729 (U.S. & Canada)
817.640.7666
Fax: 817.640.6644

PHILIPS

DUNLEE

Visit us at www.dunlee.com

DUNLEE